Tubes Radiogènes à Anode Tournante  
Drehanoden - Röntgenröhre  
Tubos de Rayos-X con Ánodo Giratorio

Note: Document originally drafted in the English language.

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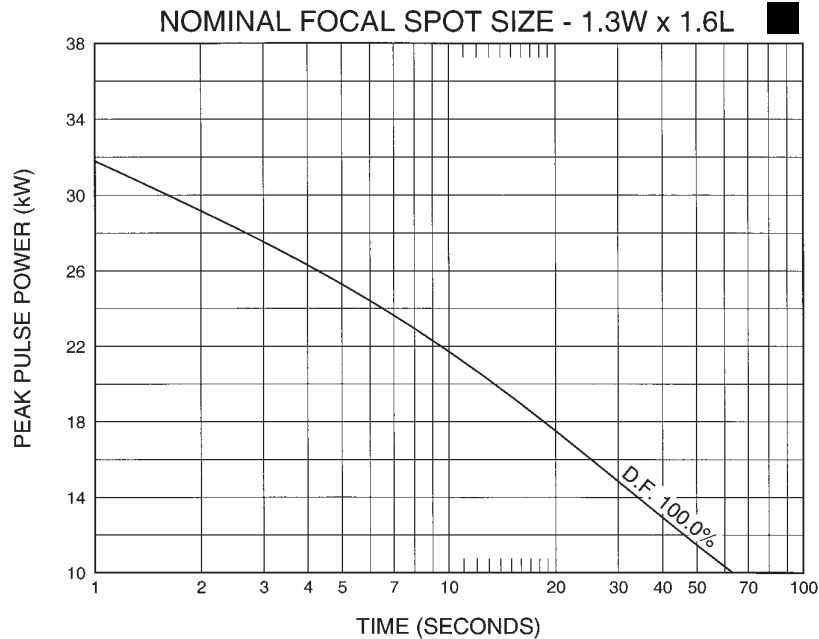
Manufactured by Varian Medical Systems  
Fabrique par Varian Medical Systems  
Hergestellt von Varian Medical Systems  
Fabricado por Varian Medical Systems

Specifications subject to change without notice.  
Spécifications susceptibles d'être modifiées sans préavis.  
Technische Daten ohne Gewähr.  
Especificaciones sujetas a cambio sin previo aviso.


**3 Ø Constant Potential** 

Abaques de Expositions CEI 60613  
Belastungskurven IEC 60613  
Diagramas de Exposición IEC 60613

**50 Hz**

**60 Hz**

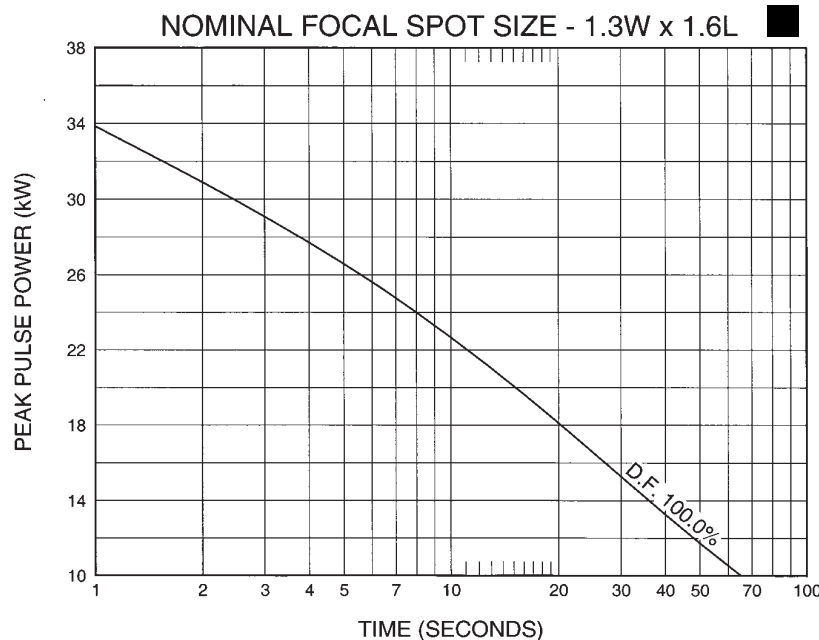**Note:**  
Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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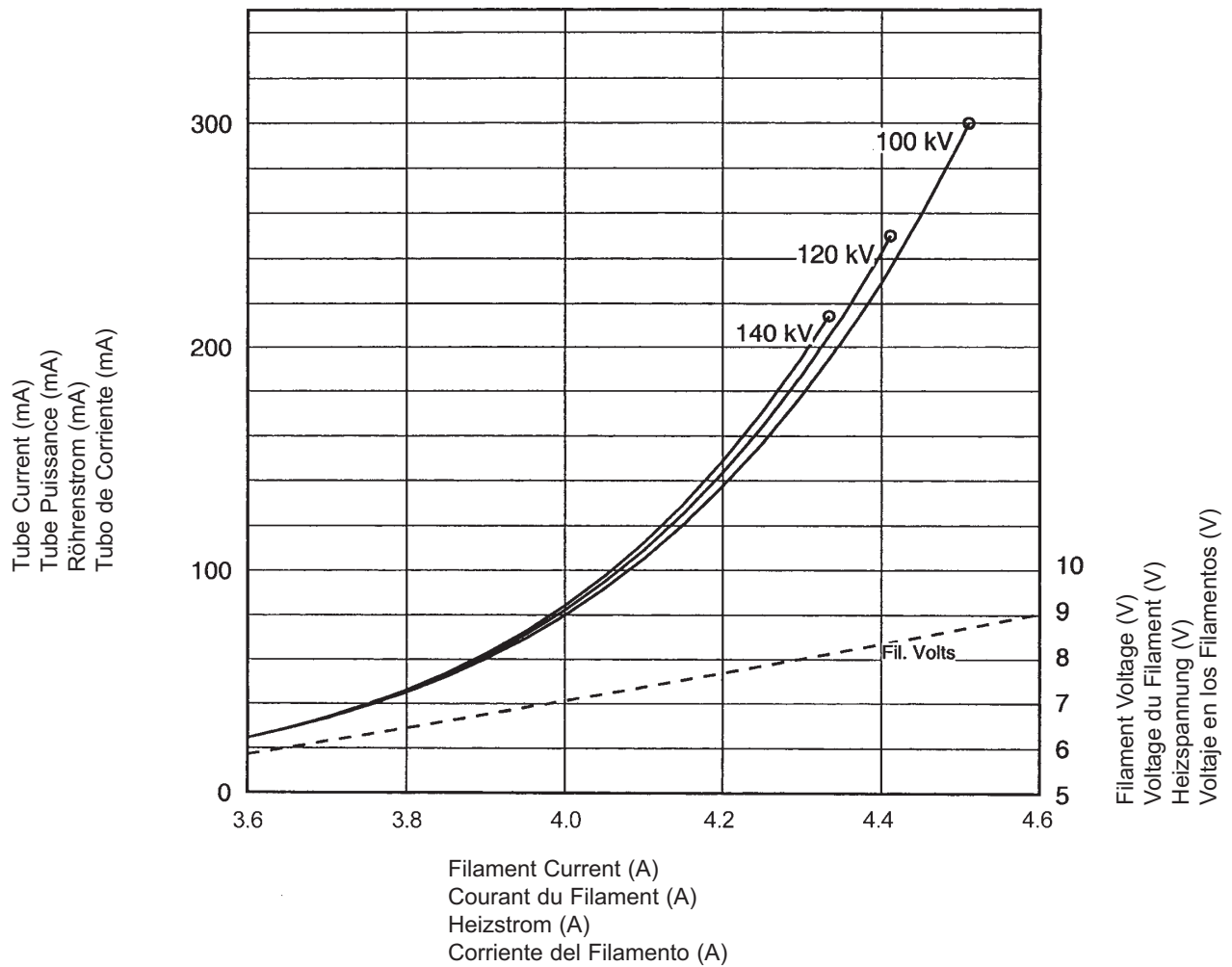
**Le Gaine B-160H**

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**Encaje de B-160H**

Maximum Peak Voltage .....	150 kV
Anode to Ground .....	75 kV
Cathode to Ground .....	75 kV
Maximum X-ray Tube Assembly Heat Content .....	1.5 MJ (2.0 MHU)
Maximum Continuous Heat Dissipation .....	1,800 W (2,520 HU/sec)
Maximum Housing Temperature .....	78°C
Maximum Heat Exchanger Dissipation .....	6,000 W (8,100 HU/sec)
Focal Point Position (Central Ray) Within 1mm (X,Y Direction from the center of radiation port.)	
X-Ray Tube Assembly	
Permanent filtration .....	1.0 mm Al IEC 60522
Loading Factors for Leakage Radiation .....	150 kV, 4.0 mA
Federal Standard High Voltage Cable .....	72
Ambient Air Temperature Limits for Operation .....	5°C to 40°C
Temperature Limits for Storage and Transport .....	-20°C to +50°C
Humidity .....	+10% to 90%
Weight: Housing .....	23 kg (50.7 lbs)
Heat Exchanger .....	12.5 kg (5.7 lbs)
IEC Classification .....	Class 1
Safety Devices - Thermal Switch	
Normally Closed Contact .....	Opening at 77°C ±3°C
Flow Switch - Normally Open Contacts	
	Contacts close with adequate oil flow.
Filament Frequency Limits .....	50 Hz - 20 kHz
Power Supply .....	DC

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
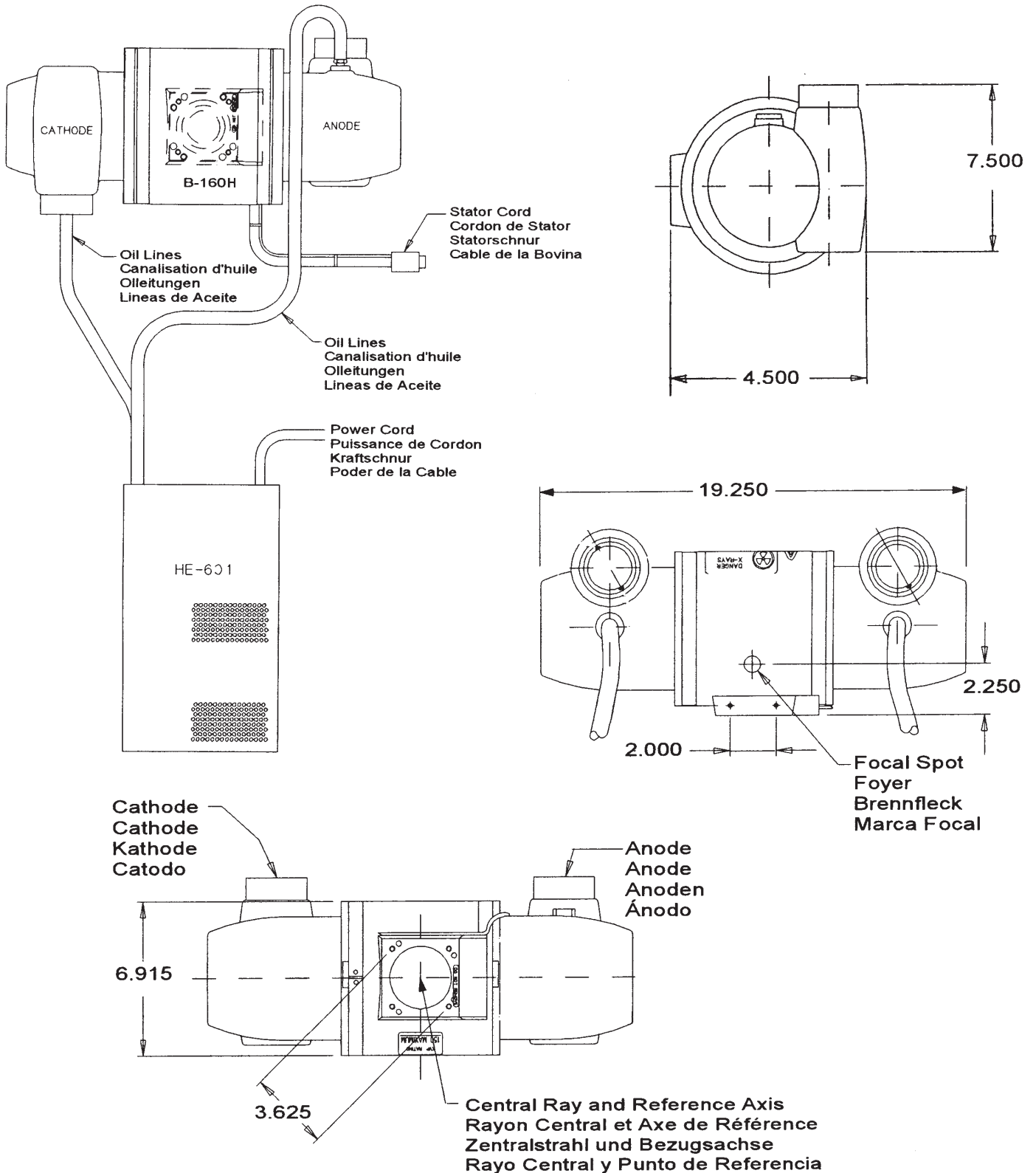
**TUBE HOUSING ASSEMBLY HEATING AND COOLING**

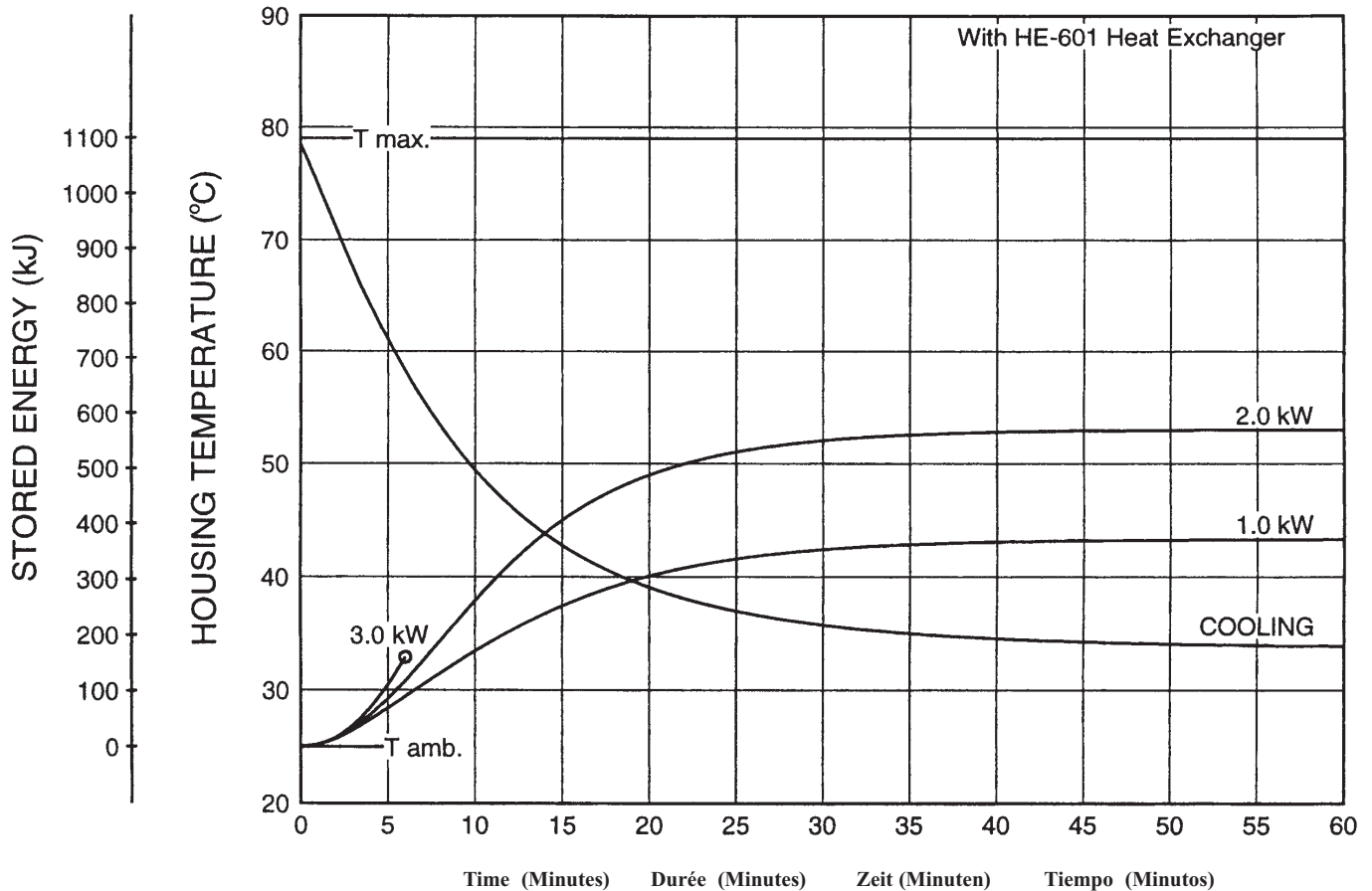**Note:**

- Heat inputs into housing include tube power, filament power, and stator power.
- Heating curves based on no restrictions of natural convection around tube housing assembly.
- Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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
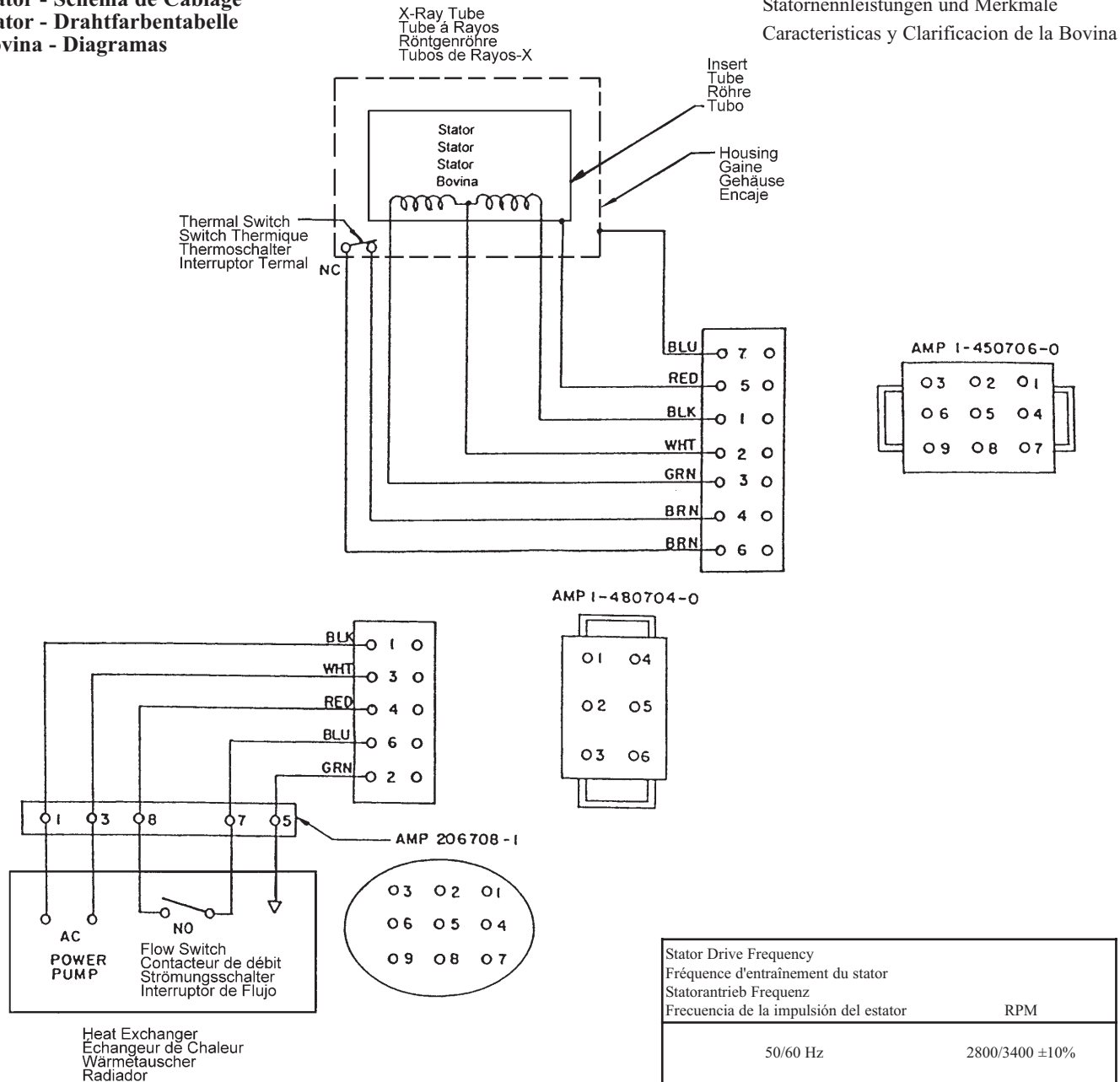
<b>Stator Supply:</b> "R" Stator		
<b>Stator Coil Resistance:</b>		
Black to White	14.0 Ohms ±15%	
Green to White	46.0 Ohms ±15%	
<b>Starter Voltage:</b>	<b>Start</b>	<b>Run</b>
50/60 Hz	220 VAC	60 VAC
<b>Time to Full Speed:</b>		
50/60 Hz	0 - 2700 RPM	2 Sec.
<b>X-Ray Tube Assembly:</b>		
GS-1089/B-160H	IEC 60601-2-28	

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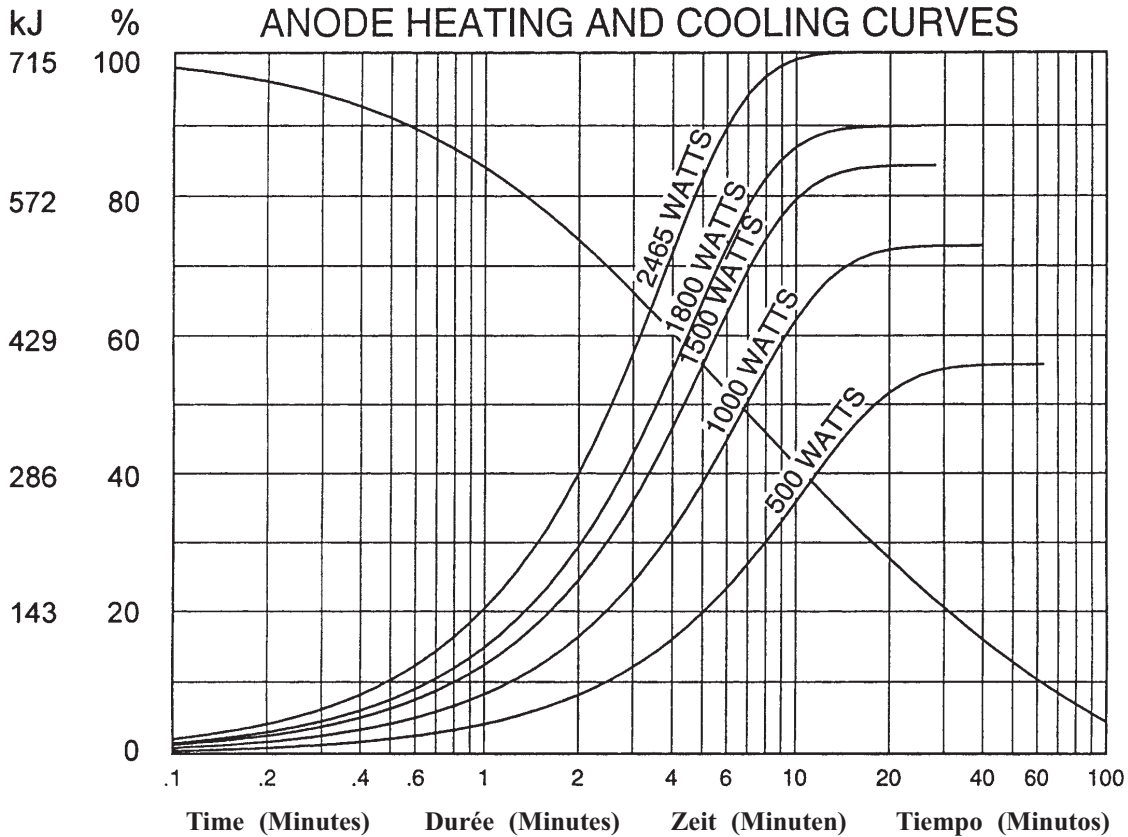**Note:**  
1. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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