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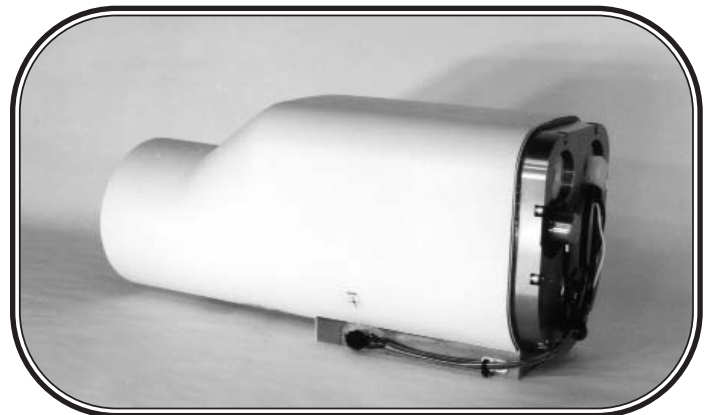
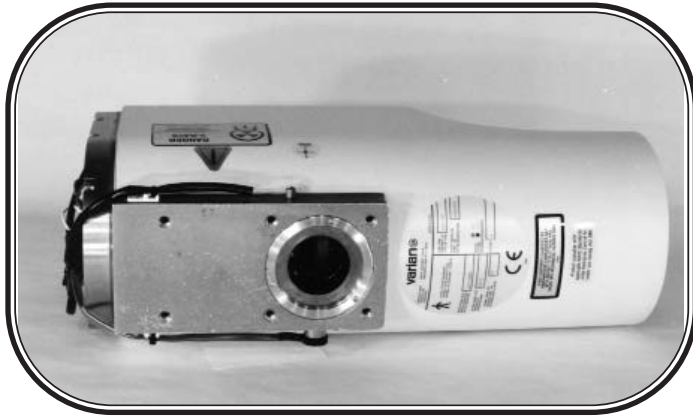
Note: Document originally drafted  
in the English language.

**Product Description:**

The ORIII is a uniquely designed x-ray tube housing developed for use in surgical C-Arm applications. This design utilizes a high performance rotating anode tube, with inherently greater target loading capabilities than that of stationary anode tube designs. This housing provides ease of use at a nominal patient table height.

**Housing:** Shockproof, Aluminum. Lead lined; filled under vacuum with high dielectric insulating oil and hermetically sealed. The housing contains a stator for driving the rotor.

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Manufactured by Varian Medical Systems  
Fabrique par Varian Medical Systems  
Hergestellt von Varian Medical Systems  
Fabricado por Varian Medical Systems

Specifications subject to change without notice.  
Spécifications susceptibles d'être modifiées sans préavis.  
Technische Daten ohne Gewähr.  
Especificaciones sujetas a cambio sin previo aviso.

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Maximum Voltage	
Anode to Cathode .....	125 kV
Anode to Ground .....	62.5 kV
Cathode to Ground .....	62.5 kV
Maximum Housing Cooling Rate .....	185 W (15 kHU.**/Min.)
Maximum Housing Temperature .....	78°C
X-ray tube assembly cooling provided by ambient air flow.	
Thermal Characteristics	
Housing heat storage capacity .....	1,185 kJ (1.6 MHU**)
Weight .....	19.5 Kg (43 lbs.) without cables or accessories
Finish	
Textured light grey color is standard. Other colors are also available.	
X-Ray Tube Assembly	
Permanent Filtration .....	0.7mm Al minimum at 75 kV IEC 60522
Loading Factors for Leakage Radiation .....	125 kV, 1.5 mA
X-Ray Coverage	
8.5° target - 12 in (30 cm) at 40 in (100 cm) Distance Source-Image	
10° target - 14 in (35 cm) at 40 in (100 cm) Distance Source-Image	
Thermal Switch	
The switch must be used in conjunction with the control manufacturer's interlock circuitry. The thermal switch is normally closed. The switch will open when the temperature reaches 85°C ±2.8°C.	
CAUTION	
The thermal switch does not protect the insert against overheating.	
IEC Classification .....	Class 1
**H.U. (Heat Units) = (kV) x (mA) x 1.35 x Time in seconds (3Ø operation)	

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
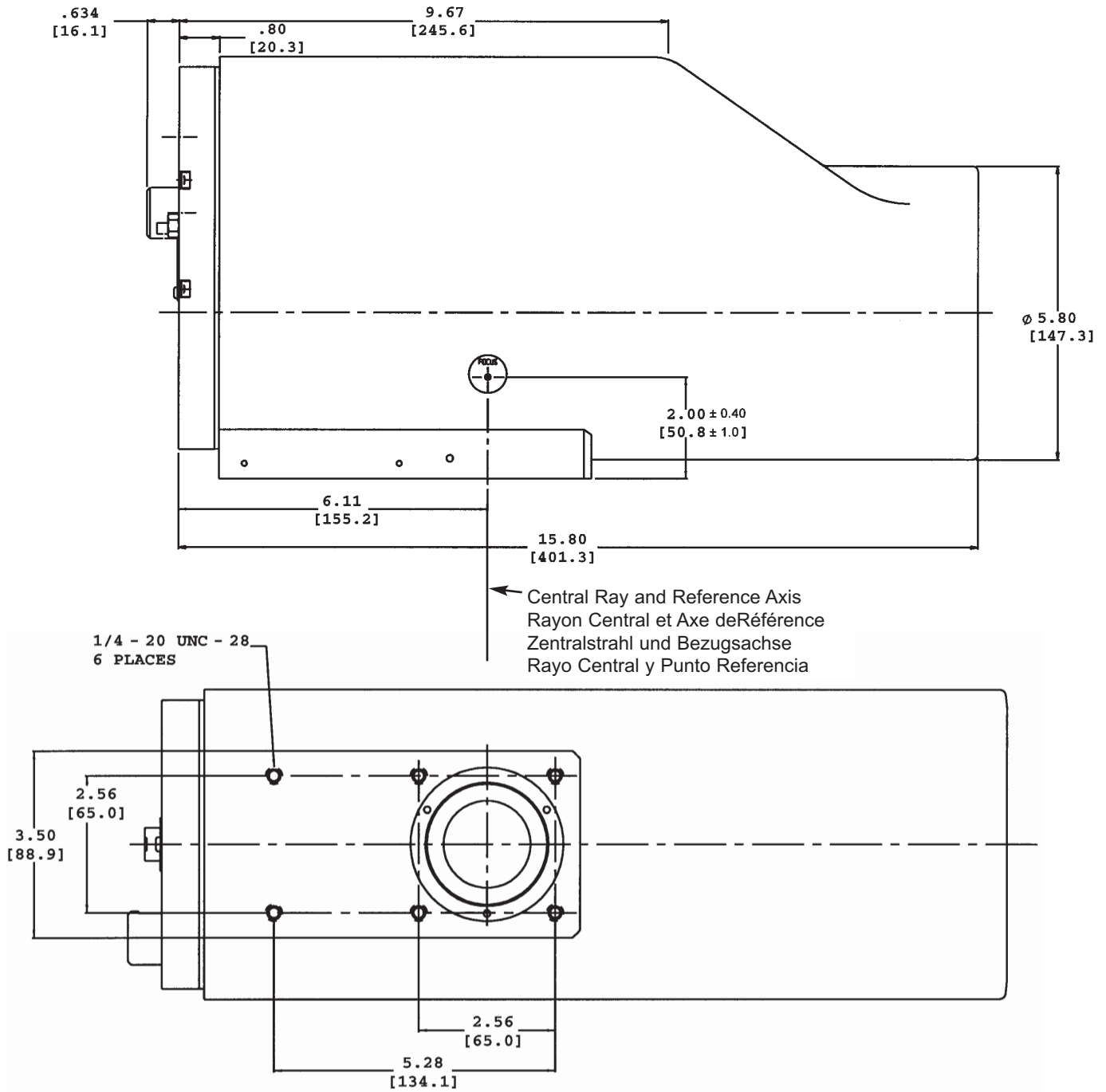
**INCHES**  
**[MM]**

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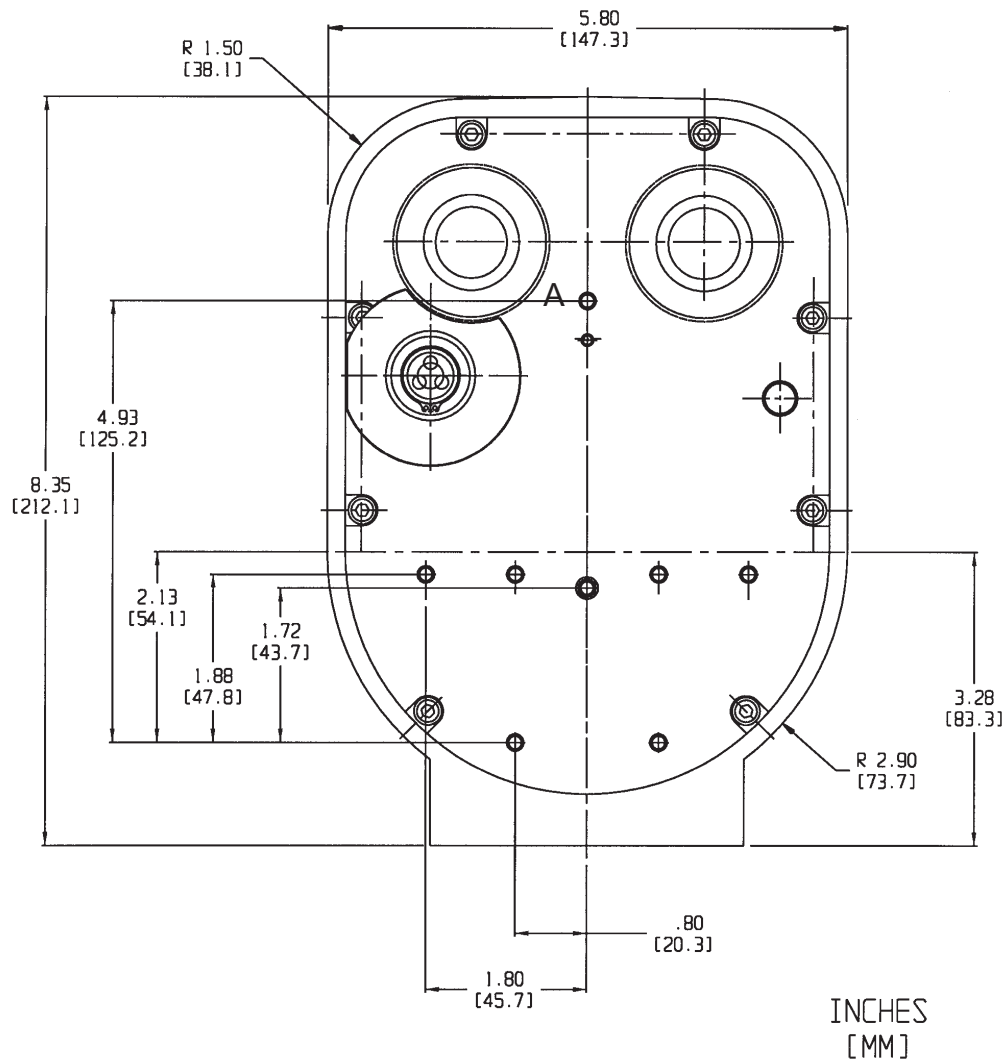Esquema Dimensional

**Cable Receptacles:** Alden type, P702-1 (cathode) and P702 (anode), high voltage receptacles. The receptacles mate with Alden type (P7010S) high voltage connector and cable.

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**Rayproofing:** Stray radiation complies with DHHS standards under the "Radiation Control for Health and Safety Act 1968" (U.S.A.)

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**Stator:** The ORIII housing is equipped with a standard Varian stator requiring a 30 microfarad capacitor, providing 90° phase shift for 50/60 Hz., 2,850/3,450 rpm operation.

<b>Stator Motor Ratings:</b>	<u>Frequency (Hz.)</u>	<u>Volts</u>	<u>Common Leg Current</u>
	50-60	60*	Approx. = 2.0 Amps
	50-60	120	Approx. = 4.0 Amps
	50-60	240	Approx. = 8.0 Amps

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**Stator Cord:** A six wire shield-grounded stator cord should be connected from the stator terminal to the motor control. All listed voltage and amperage measurements should be taken at the housing end.

If the stator cord is not provided, the cord used must be UL listed or a UL recognized component for all products used in the USA. Outside the USA, the cord used must comply with all applicable regulatory and statutory requirements for electrical and safety. The cord must have the following characteristics: six conductor, 18AWG (0.9mm<sup>2</sup>) Cu, rated 600V, 90°C minimum, shielded cable with PVC jacket 0.032 inch (0.081mm) thick minimum. Lead terminations are provided and must be used with wire that matches the above description.

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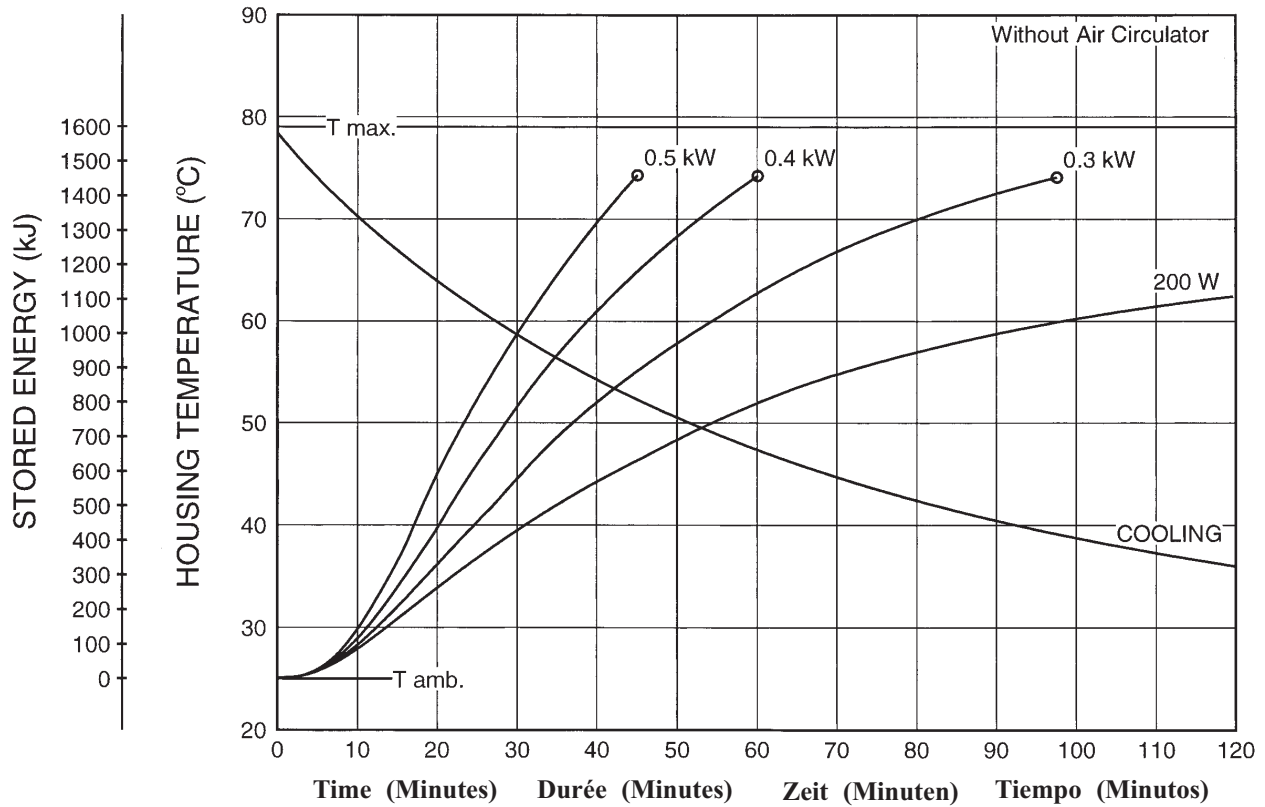
### X-ray Tube Assembly Heating and Cooling Curve

**Operating Temperature:** Minimum ambient temperature for storage and transportation is  $-18^{\circ}\text{C}$  ( $0^{\circ}\text{F}$ ). Operating range of the ORIII housing is  $16^{\circ} - 72^{\circ}\text{C}$  ( $60^{\circ} - 167^{\circ}\text{F}$ ).

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Les pertes en radiation sont conformes aux normes DHHS du "Radiation Control for Health and Safety Act 1968" (U.S.A.).
Der Strahlenverlust entspricht den Vorschriften des DHHS "Radiation Control for Health and Safety Act 1968" (U.S.A.).