Note: Document originally drafted in the English language.

Product Description

The B-150H housing is designed for Varian rotating anode inserts having 102mm (4 inch) diameter targets.

IEC Classification Class 1

Weight, Approximate:

Housing 19.5 kg (43 lbs)
with Trunnion Rings 22.9 kg (50.5 lbs)
with Trunnion Rings and Fan Assembly .. 24.1 kg (53.2 lbs)

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Soporte Entrada ó de Rotable

Cable de la Bovina 0.9 m (3 ft)

Manufactured by Varian Medical Systems
Fabrique par Varian Medical Systems
Hergestellt von Varian Medical Systems
Fabricado por Varian Medical Systems

Specifications subject to change without notice.
Spécifications susceptibles d’être modifiées sans préavis.
Technische Daten ohne Gewähr.
Especificaciones sujetas a cambio sin previo aviso.

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Produktbeschreibung

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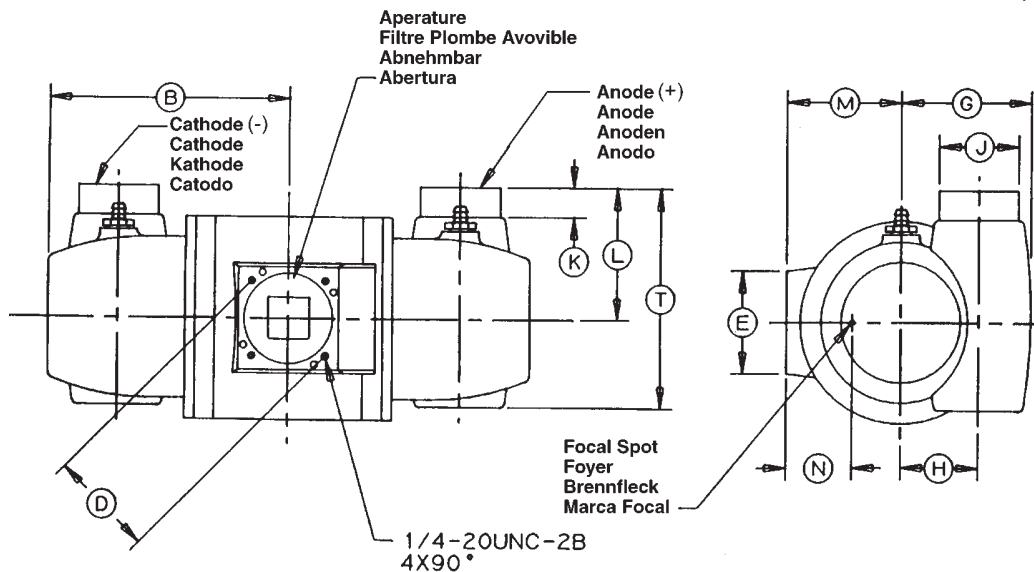
| | |
|------------------------------------------------------------------------|--------------------------|
| Nominal X-ray Tube Voltage | 150kV |
| Maximum Cathode to Ground | 75 kV |
| Maximum Anode to Ground | 75 kV |
| Grid to Cathode (If applicable) | -4 kV |
| Grid Control Voltages | |
| Typical Bias Voltage for Cutoff at 150 kV | -3600 Vdc |
| Grid Voltage for Exposure | 0 Vdc |
| Maximum X-Ray Tube Assembly Heat Content .. | 1,111 kJ (1,500 kHU) |
| Maximum Continuous Heat Dissipation (max. housing temperature 78°C) | |
| without air circulator | 200 W (270 HU/sec) |
| with air circulator | 400 W (540 HU/sec) |
| X-Ray Tube Assembly | |
| Permanent Filtration | 0.7mm Al/75 kV IEC 60522 |
| Loading Factors for Leakage Radiation | 150 kV, 4.0 mA |
| Ambient Air Temperature Limits for Operation | 5°C to 40°C |
| Temperature Limits for Storage and Transport | |
| Humidity | -20°C to +75°C |
| Atmospheric Pressure Range | +10% to +90% |
| | 70 kPa to 106 kPa |
| X-Ray Tube Assembly.(Complies to) | IEC 60601-2-28 |

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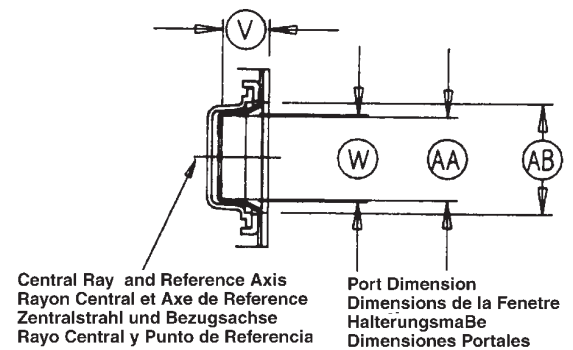
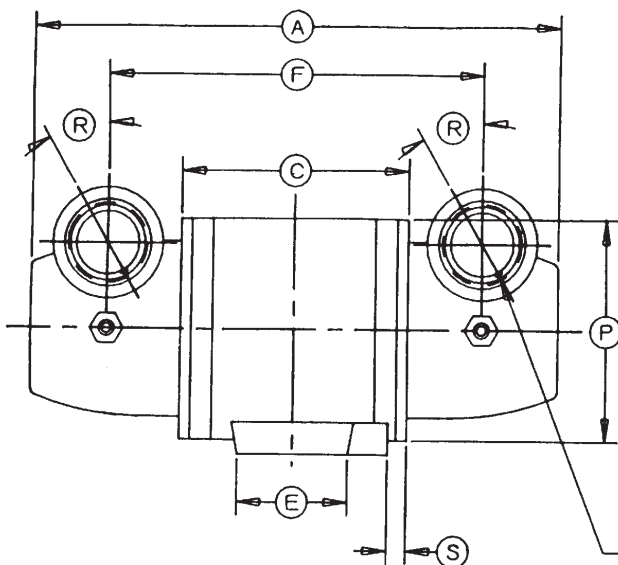
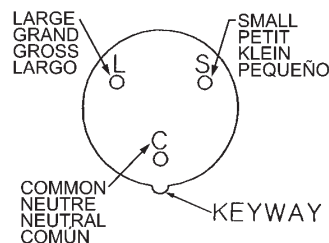
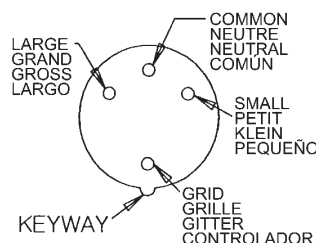
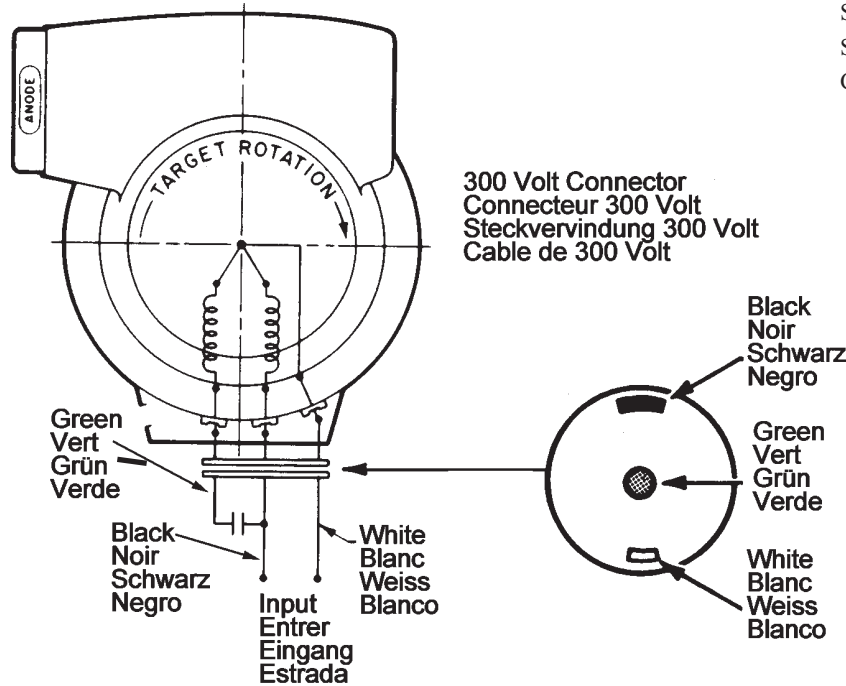
TO ENSURE PROPER TARGET ROTATION, EACH STATOR CORD IS EQUIPPED WITH A SPECIAL TWIST-LOCKING CONNECTOR AND A MATING CONNECTOR.

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Stator Power:

Time to full speed of the anode is a function of the power rating of the "starter" and the weight / diameter of the anode. All Varian stator types are rated for regular speed and high speed starters. Time to full speed of 4 inch (102mm) anode series tubes is between 1.3 and 2.0 seconds.

Immediately following high speed anode rotation, the rotor speed must be reduced to 4000 r/min or less within 10 seconds using a suitable dynamic braking device.

No more than two high speed starts per minute are permissible. The starting voltage must never exceed 500 volts rms.

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