Tubes Radiogènes à Anode Tournante
Drehanoden - Röntgenröhre
Tubos de Rayos-X con Ánodo Giratorio

GS-1580/B-180H

Note: Document originally drafted in the English language.

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

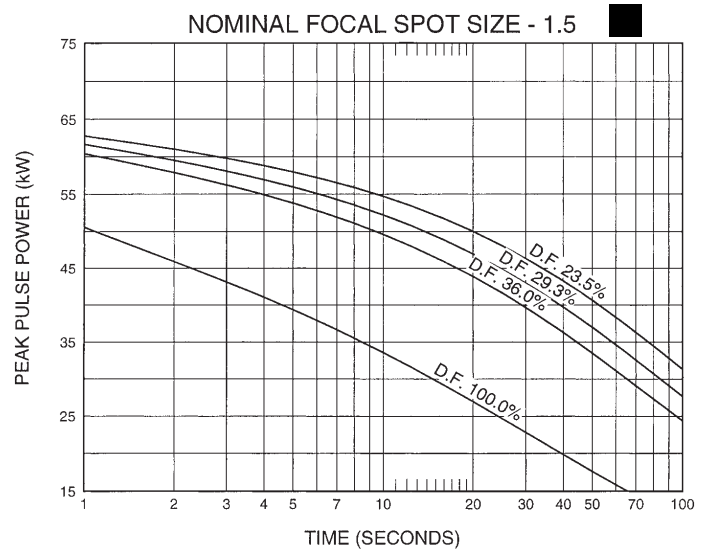
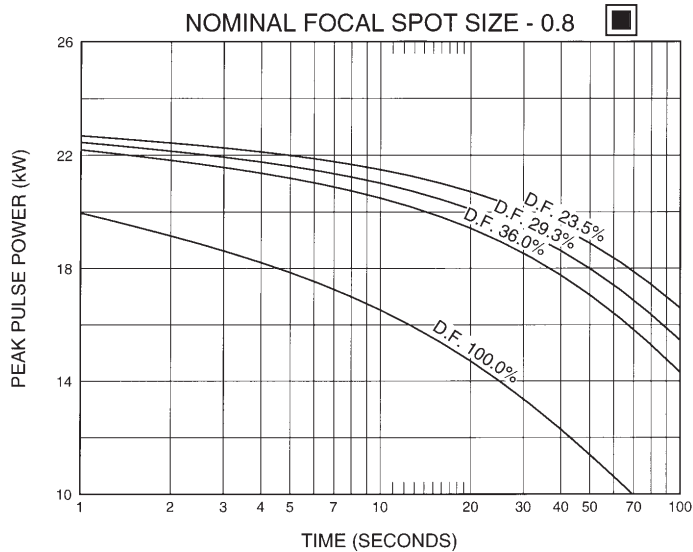
50/60 Hz

150/180 Hz

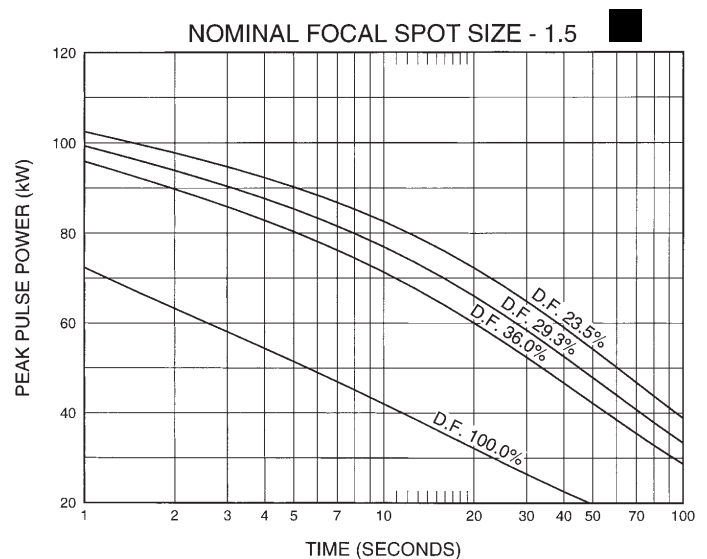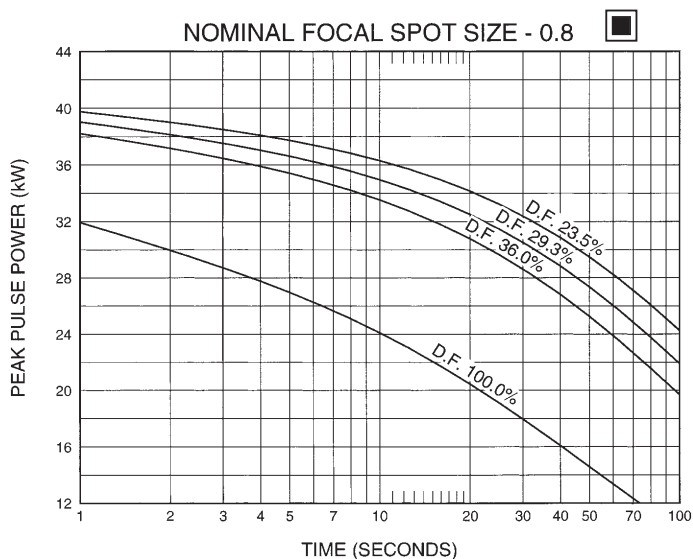Note:
Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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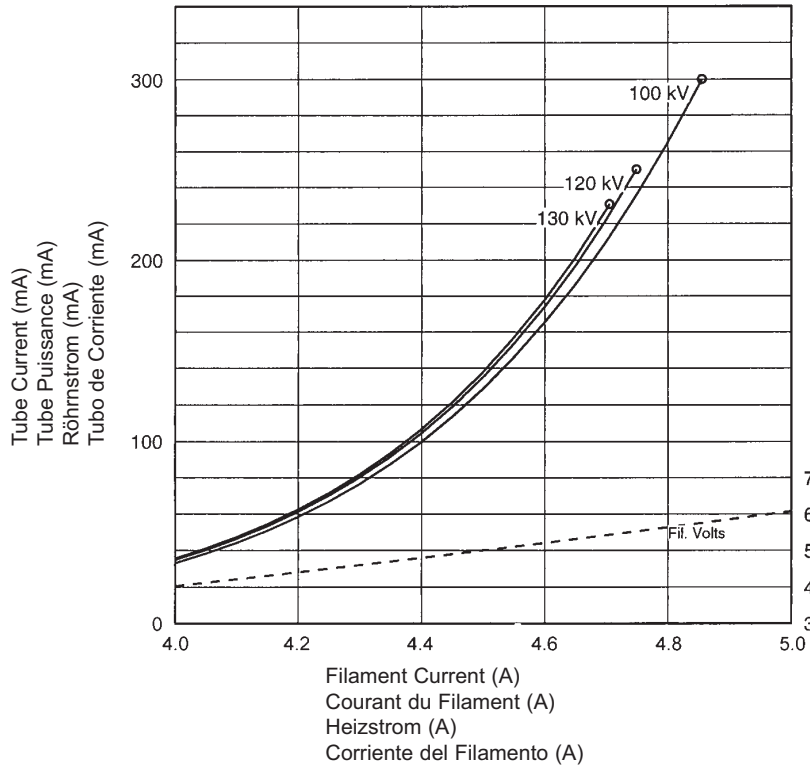
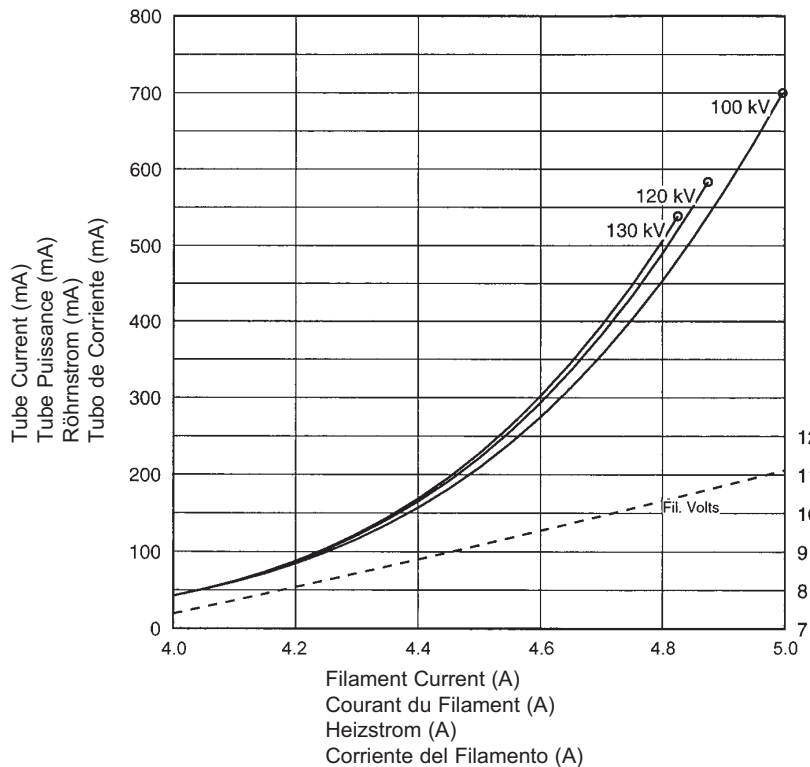
Le Gaine B-180H

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Encaje de B-180H

Maximum Peak Voltage	150 kV
Anode to Ground	75 kV
Cathode to Ground	75 kV
Maximum X-ray Tube Assembly Heat Content	1,430 kJ (2.0 MHU)
Maximum Housing Temperature	78°C
Maximum Continuous Heat Dissipation (Includes stator heat)	2.9 kW (4,060 HU/sec)
Maximum Heat Exchanger Dissipation	5.0 kW (7,050 HU/sec)
Focal Point Position (Central Ray) Within 1mm (X,Y Direction from the center of radiation port.)	
X-Ray Tube Assembly Permanent filtration	1.0 mm Al IEC 60522
Loading Factors for Leakage Radiation	150 kV, 4 mA
Federal Standard High Voltage Cable	72
Ambient Air Temperature Limits for Operation	5°C to 40°C
Temperature Limits for Storage and Transport	-20°C to +70°C
Humidity	+10% to +90%
Weight: Housing & Tube	29.0 kg (63.9 lbs)
Heat Exchanger	16.0 kg (35.3 lbs)
IEC Classification	Class 1
Safety Devices - Thermal Switch Normally Closed Contact	Opening at 76.7°C ±2.8°C (170°F ±5°F)
Rating	3A @ 120Vac or 30 Vdc max.
Flow Switch - Normally Open contact	Contacts close with adequate oil flow
Filament Frequency Limits	0 - 400 Hz
Power Supply	DC

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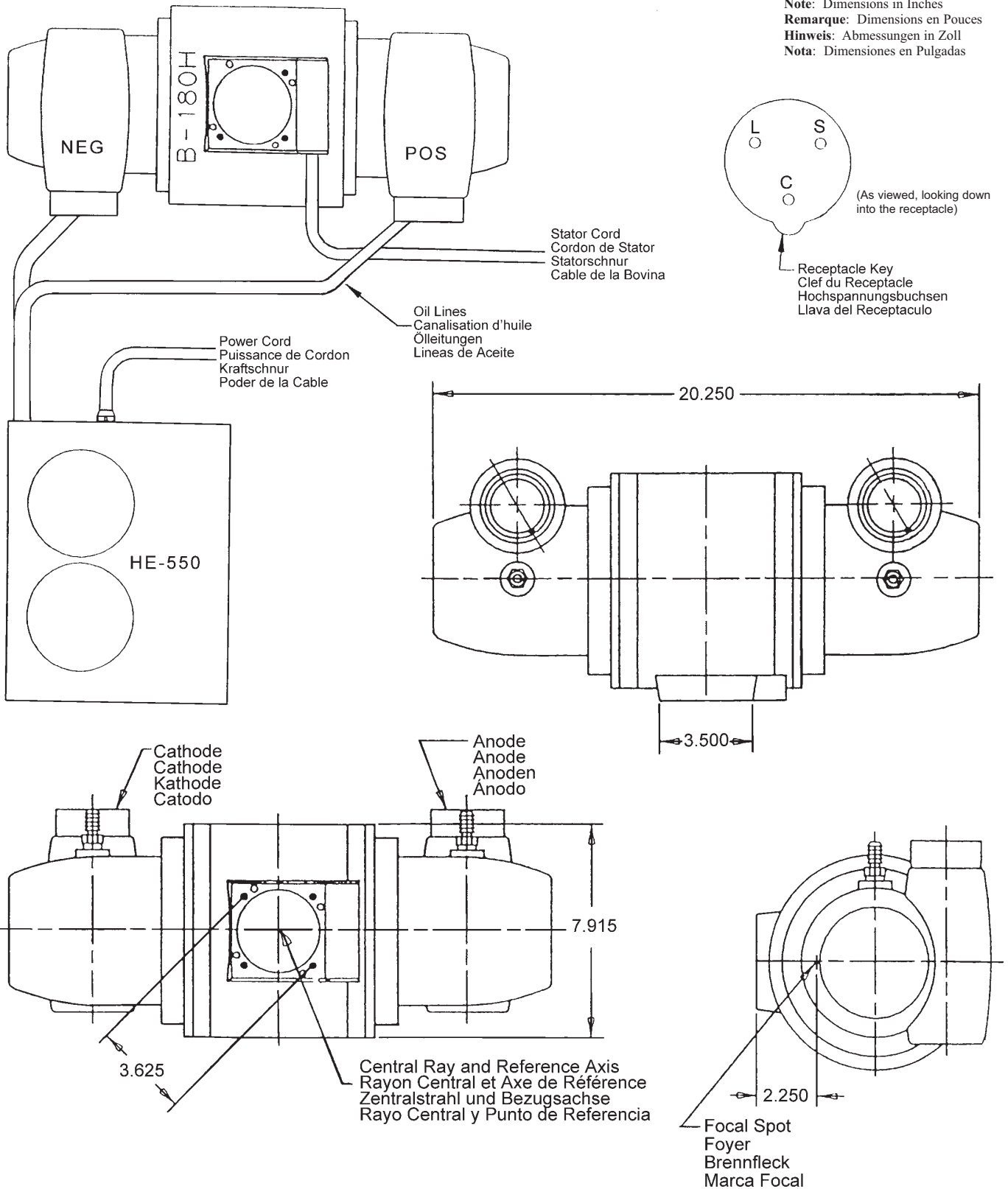
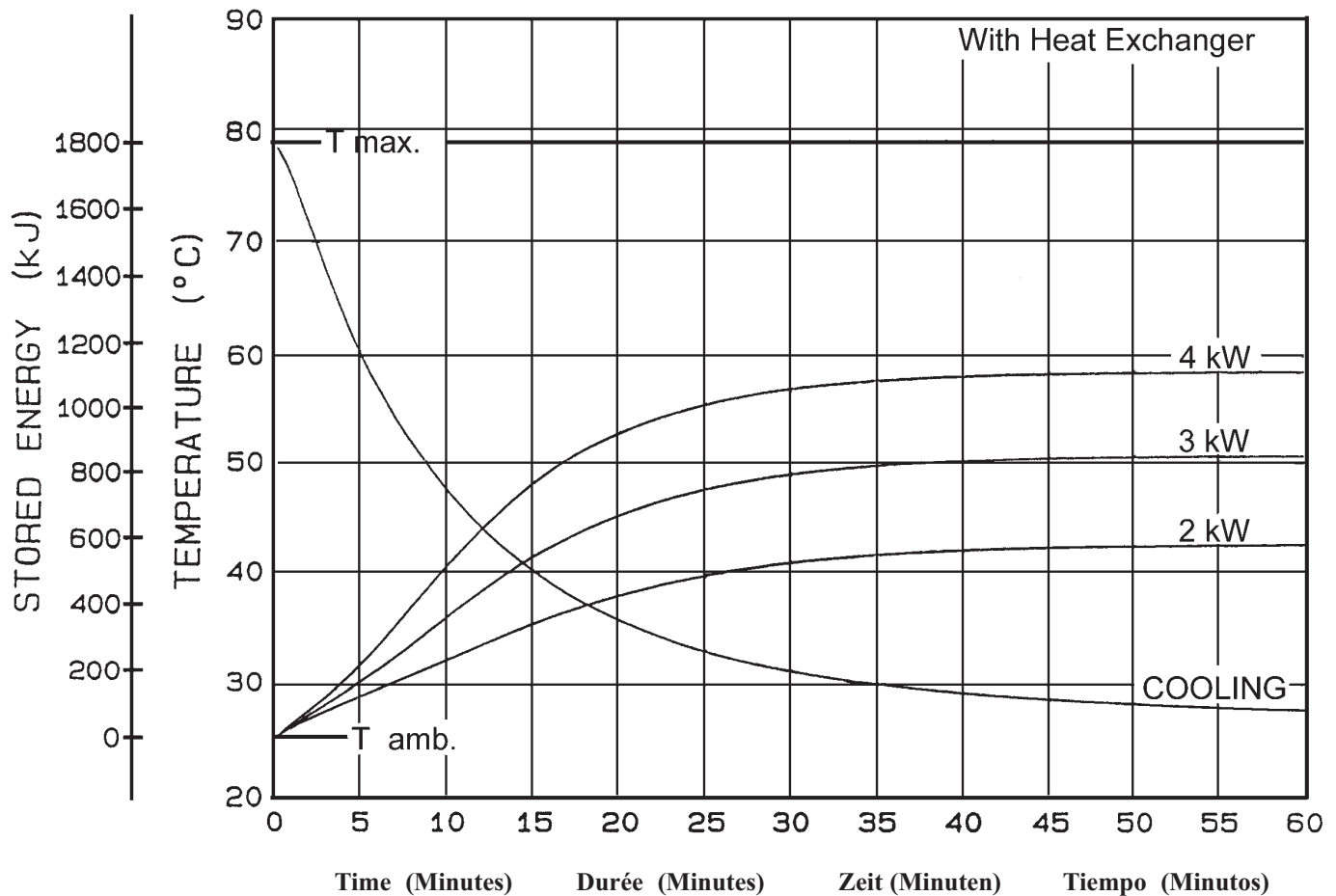
TUBE HOUSING ASSEMBLY HEATING AND COOLING

Note:
1. Heat inputs into housing include tube power, filament power, and stator power.
2. Heating curves based on no restrictions of natural convection around tube housing assembly.
3. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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
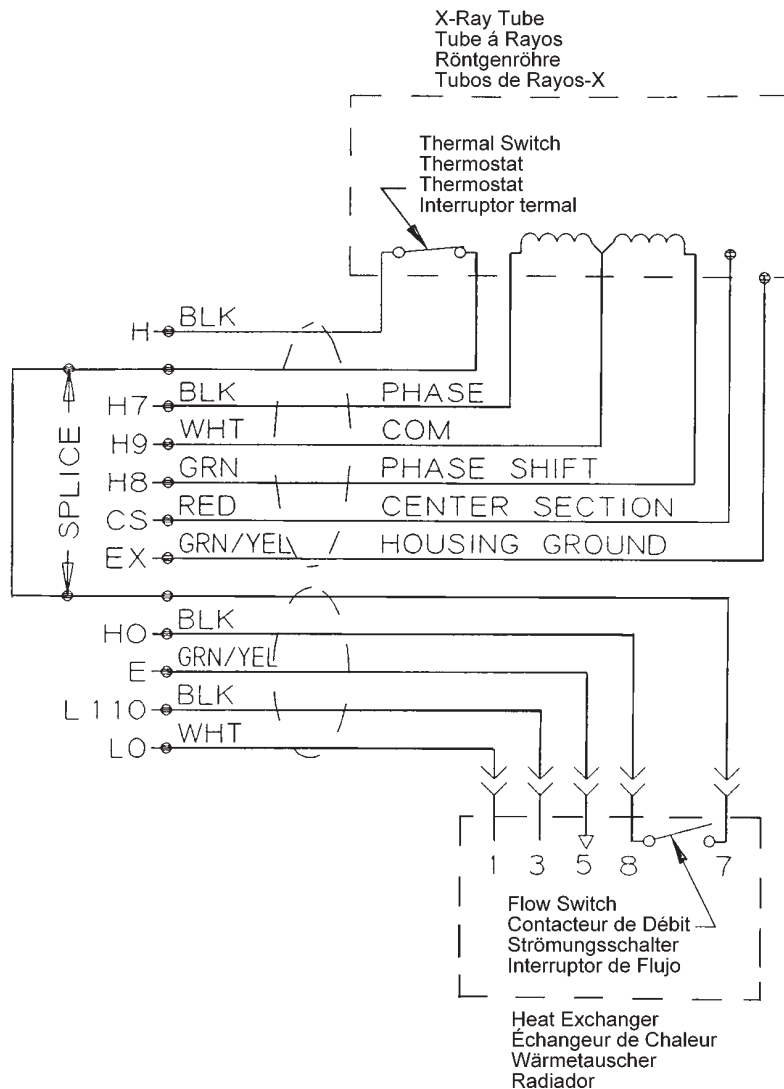
Stator Type: "R" Stator		
Stator Coil Resistance:		
Black to White	14.0 Ohms ±15%	
Green to White	46.0 Ohms ±15%	
Starter Voltage:		
	Start	Run
50/60 Hz	265 VAC	75 VAC
150/180 Hz	600 VAC	115 VAC
Time to Full Speed:		
50/60 Hz	0 - 2700 RPM	6.3 Sec.
150/180 Hz	0 - 8000 RPM	6.3 Sec.
X-Ray Tube Assembly:		
GS-1580/B-180H	IEC 60601-2-28	

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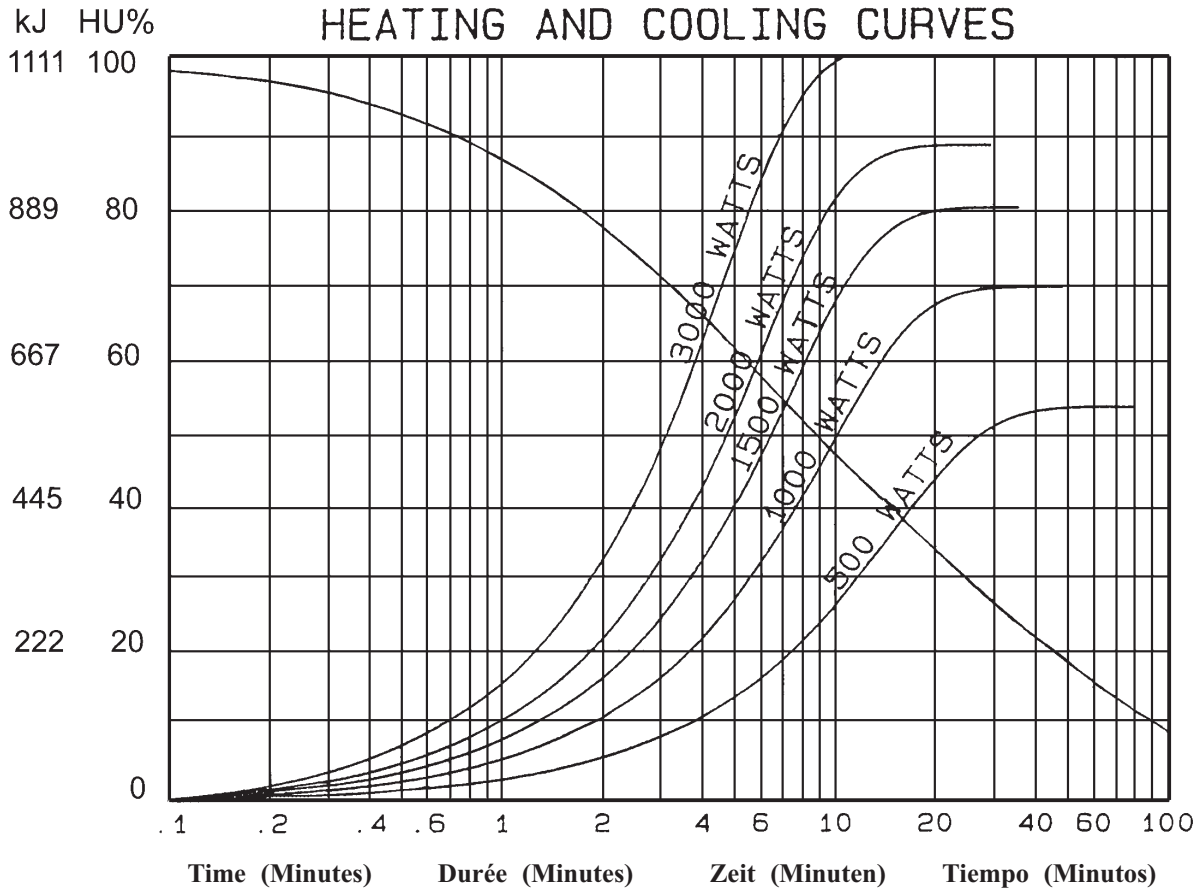Note:
Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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