Product Description

The MX100-18™ housing is used for rotating anode inserts having 100mm (4 inch) diameter targets.

IEC Classification Class 1

Weight, Approximate:

Housing & Tube 26.3 kg (58 lbs)

Mounting Housing Bosses

All trademarks property of the respective manufacturer

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Produktbeschreibung

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Product Description	
Maximum Potential Difference	150 kV
Cathode to Ground	82 kV
Anode to Ground	82 kV
Grid to Cathode.(If Applicable)	-4 kV
Grid Control Voltages:	
Typical Bias Voltage for Cutoff at 150 kV	-3600 Vdc
Grid Voltage for Exposure	0 Vdc
Maximum X-Ray Tube Assembly Heat Content ...	1,110 kJ (1,554 kHU)
Maximum Continuous Heat Dissipation with Air Circulator	
	740 W (1,036 HU/sec)
Maximum Housing Temperature	85°C
X-Ray Tube Assembly	
Permanent Filtration	0.7mm Al IEC 60522
Loading Factors for Leakage Radiation	150 kV, 4.0 mA
Temperature Limits for Storage and Transport	
	-9°C to +70°C
Humidity	10% to 90%
Atmospheric Pressure Range	70 kPa to 106 kPa
Pressure Switch	
	Normally Closed
	1A @ 125 Vac or 30 Vdc
Open	5 PSIG (±1.0 PSIG)
Close	4 PSIG (±1.0 PSIG)
X-Ray Tube Assembly.(Complies to)	IEC 60601-2-28

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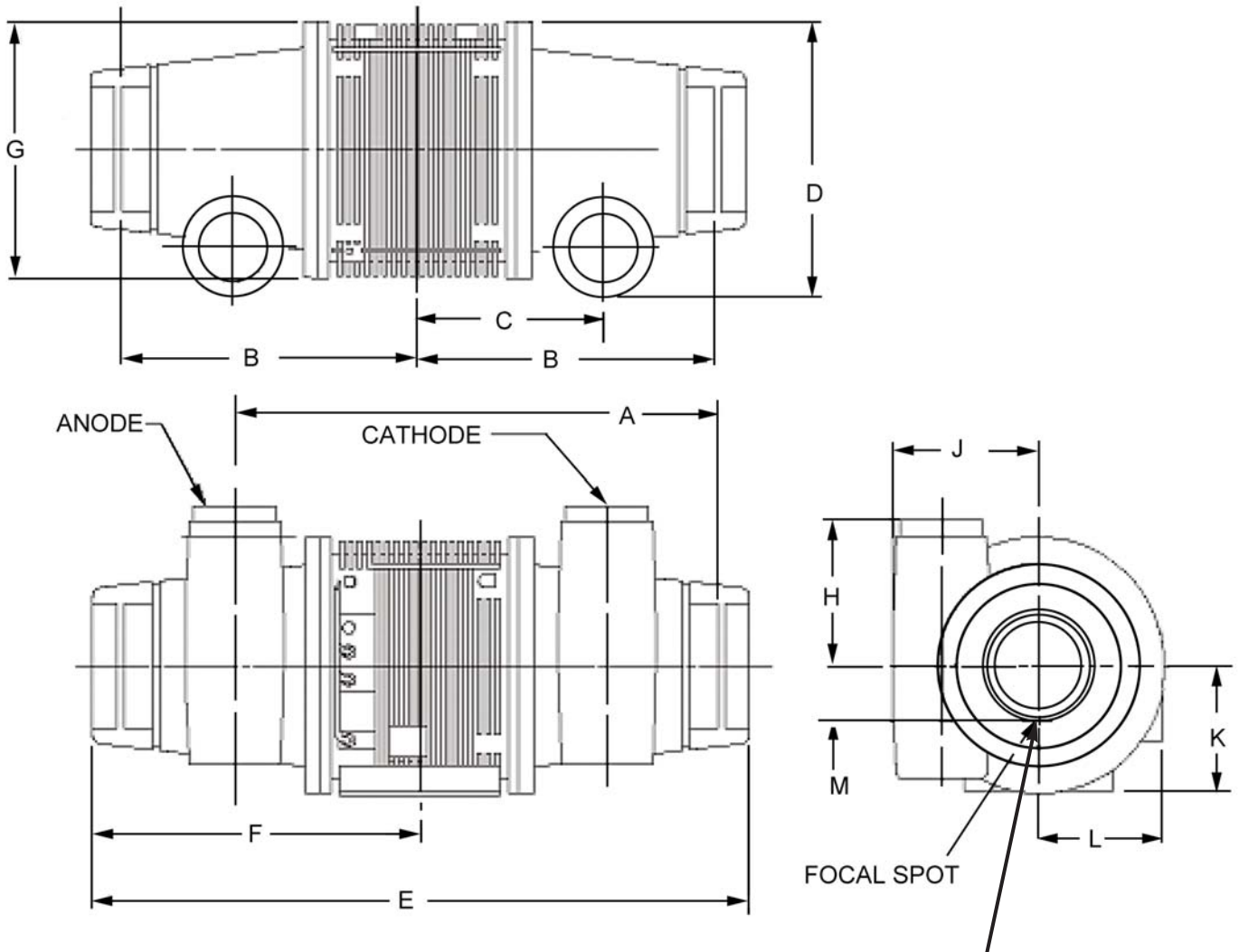
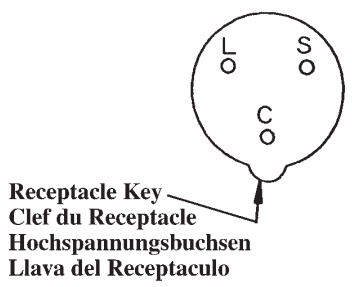
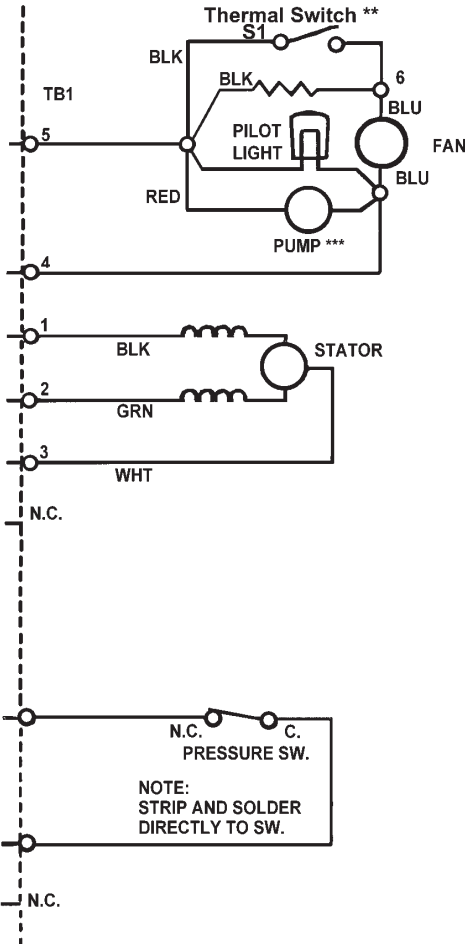
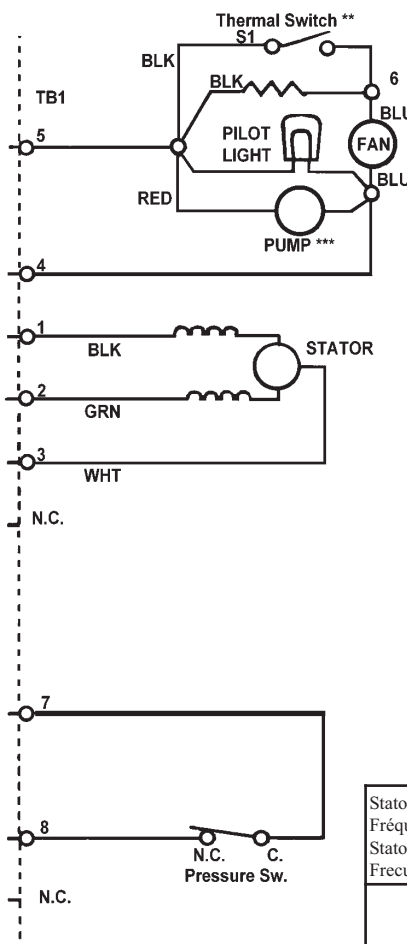
Note:
Check wiring of tube and change as necessary to make it look like above diagram.
**Used on tubes with 2 speed fans
***May not be present

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Stator Coil Resistance: 23 / 23 / 46 Ohms

Voltage:	Start	Run
60Hz	230 VAC	75 VAC
180Hz	400 VAC	85 VAC
Amperes:	Start	Run
60 Hz	6.3 A	1.5 A
180 Hz	10.3 A	2.4 A
Time to Full Speed:		
60 Hz	0-3000 RPM	.98 sec.
180 Hz	0-9000 RPM	1.33 sec.
Brake Voltage	106 VDC	
Low Speed Time	2 sec.	
High Speed Time	3.5 sec.	

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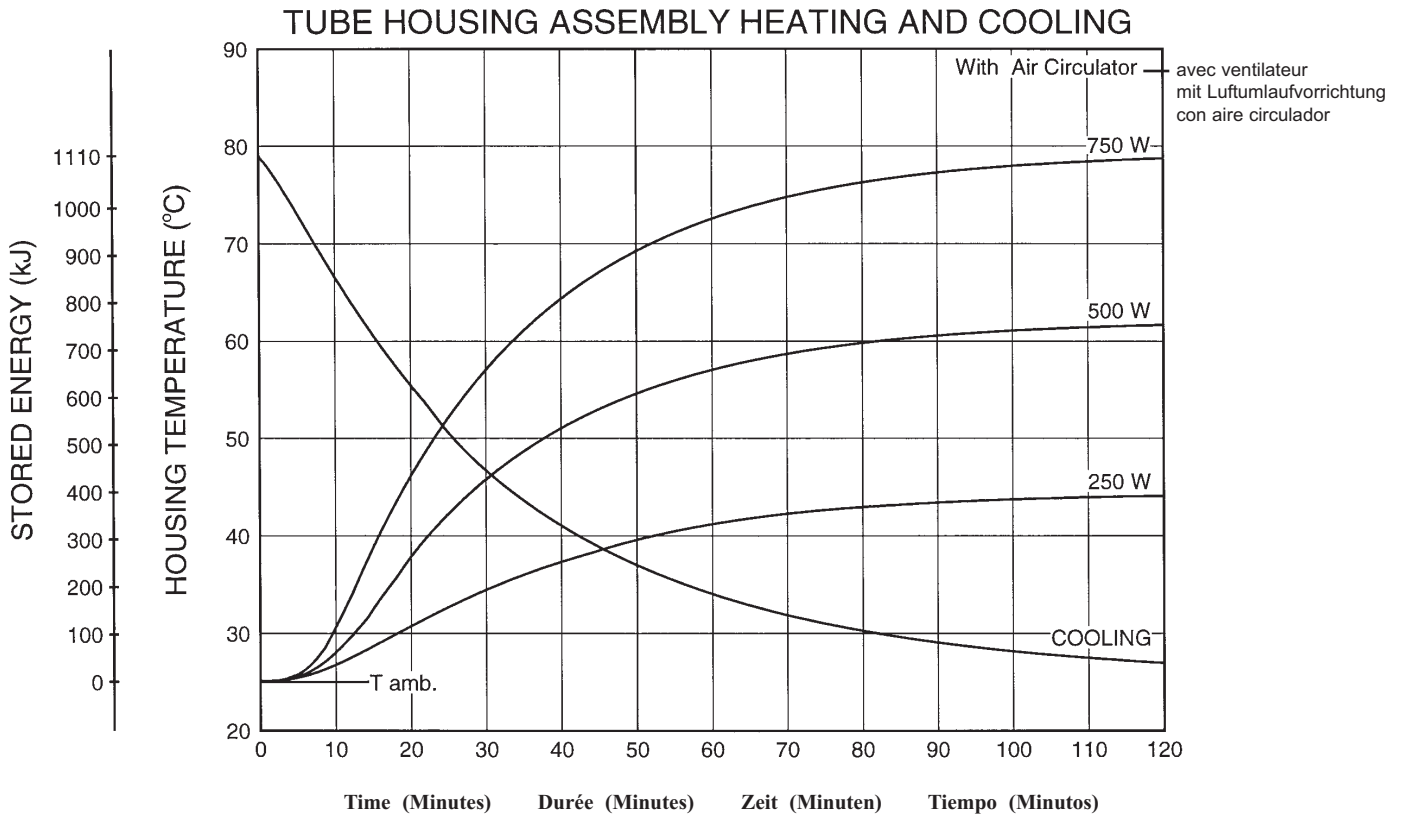
Note:
1. Heat inputs into housing include tube power, filament power, and stator power.
2. Heating curves based on no restrictions of natural convection around tube housing assembly.

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