Tubes Radiogènes à Anode Tournante  
Drehanoden - Röntgenröhre  
Tubos de Rayos-X con Ánodo Giratorio

Note: Document originally drafted in the English language.

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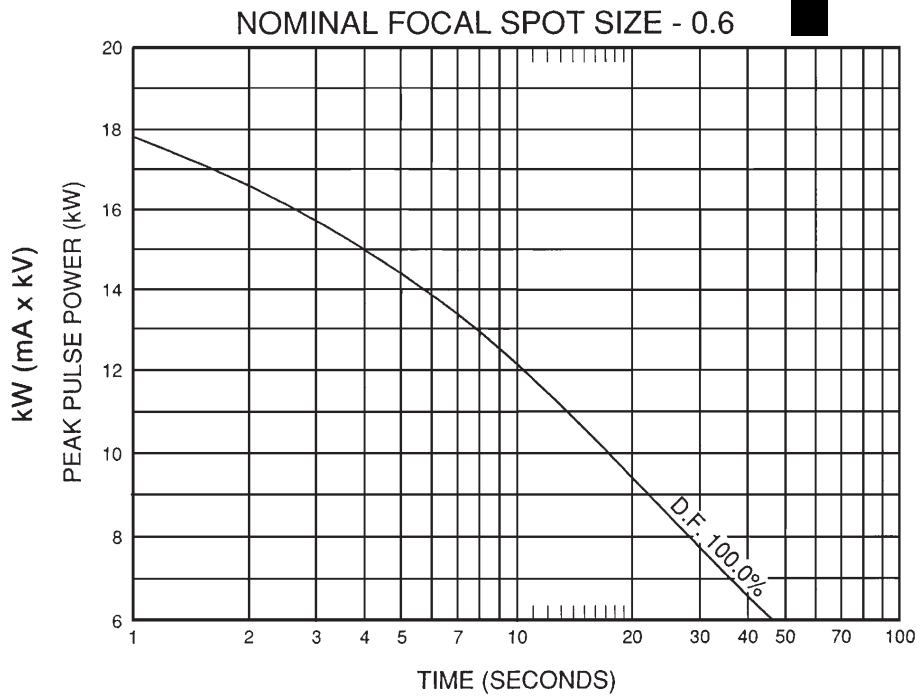
Manufactured by Varian Medical Systems  
Fabrique par Varian Medical Systems  
Hergestellt von Varian Medical Systems  
Fabricado por Varian Medical Systems

Specifications subject to change without notice.  
Spécifications susceptibles d'être modifiées sans préavis.  
Technische Daten ohne Gewähr.  
Especificaciones sujetas a cambio sin previo aviso.


**3 Ø Constant Potential** 

Abaques d'Expositions CEI 60613  
Belastungskurven IEC 60613  
Diagramas de Exposición IEC 60613

**50 Hz**


**60 Hz**

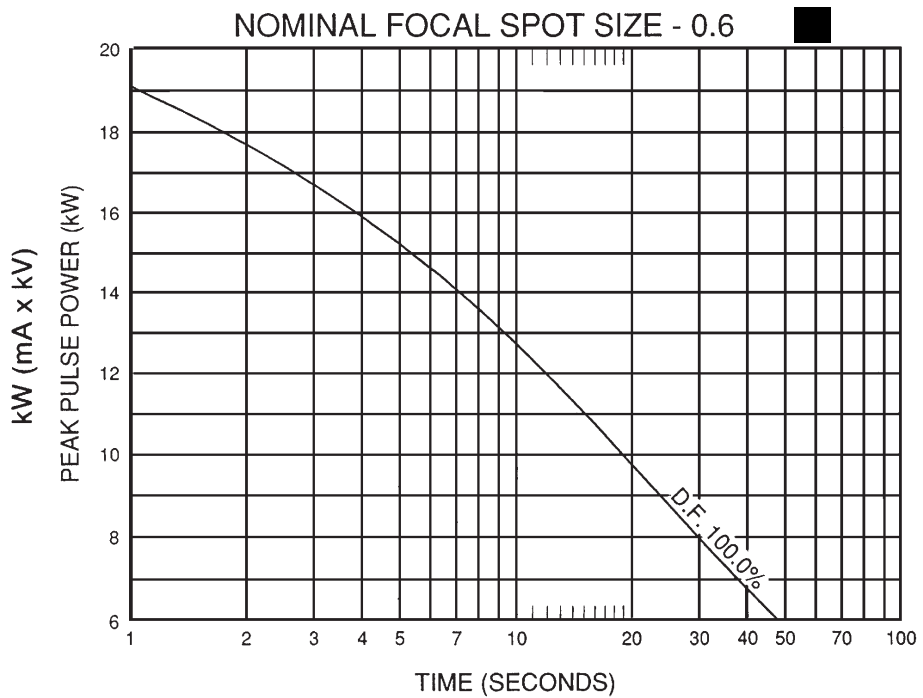3 Ø

Caractéristiques d'Émission du Filament CEI 60613  
Kathoden - Emissionskennlinien IEC 60613  
Características de Emisión del Catodo IEC 60613

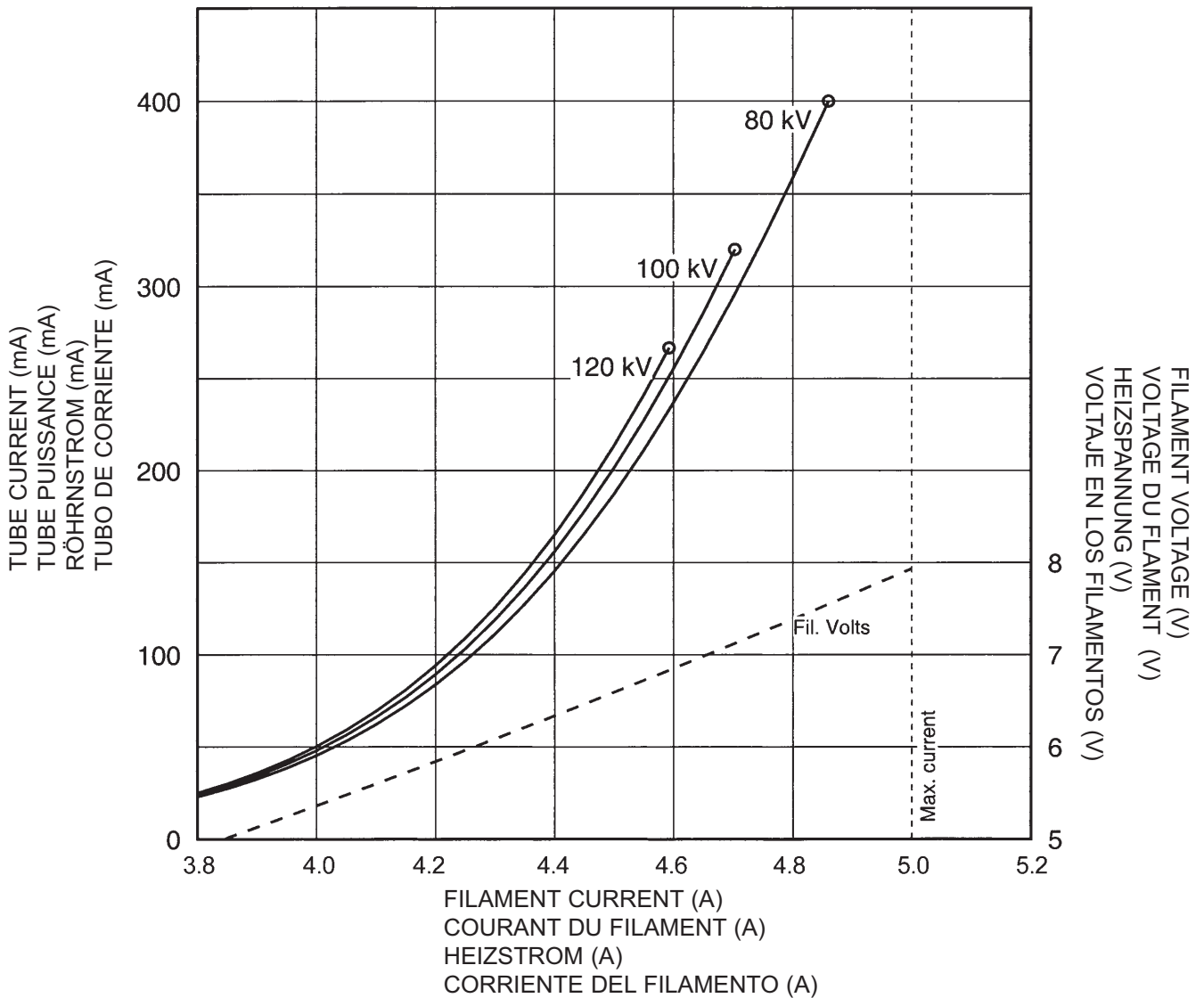**Three Phase Emission (± .15A)**  
S-270 0.6

**Le Gaine B-150H**

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**Encaje de B-150H**

<b>Product Description</b>	
Maximum Peak Voltage .....	135 kV
Anode to Ground .....	70 kV
Cathode to Ground .....	70 kV
Maximum X-ray Tube Assembly Heat Content .....	1,111 kJ (1,500 kHU)
Maximum Housing Temperature .....	78°C
Maximum Continuous Heat Dissipation .....	1,800 W (145 KHU/Min)
X-Ray Tube Assembly	
Permanent filtration .....	1.0 mm Al IEC 60522
Loading Factors for Leakage Radiation .....	135 kV, 13 mA
Federal Standard High Voltage Cable .....	72
Ambient Air Temperature Limits for Operation .....	5°C to 35°C
Temperature Limits for Storage and Transport .....	-35°C to 80°C
Weight: Housing with Heat Exchanger .....	33.5 kg (74 lb)
IEC Classification .....	Class 1
Safety Devices: Thermal Switch	
Normally Closed .....	Opening at 85°C ±3.9°C
Power Supply .....	DC

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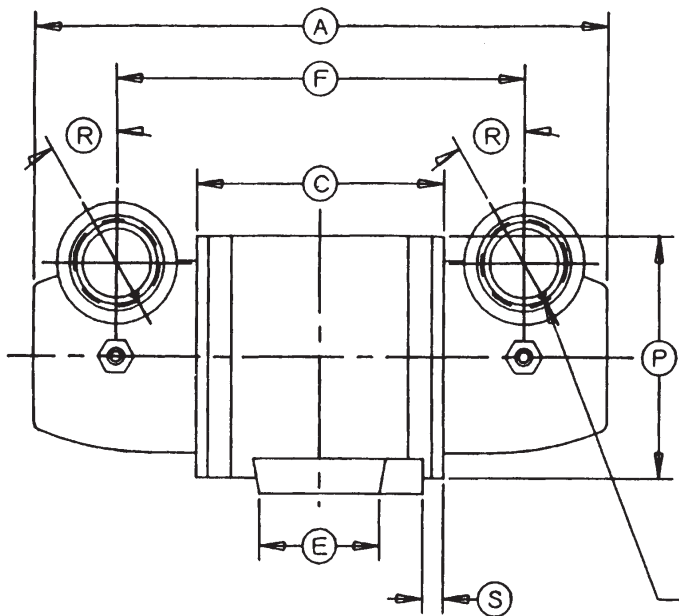
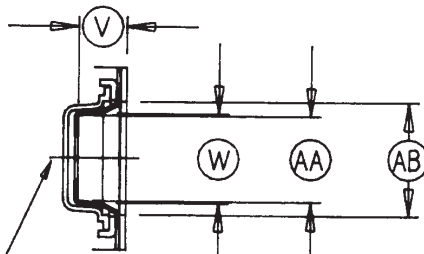
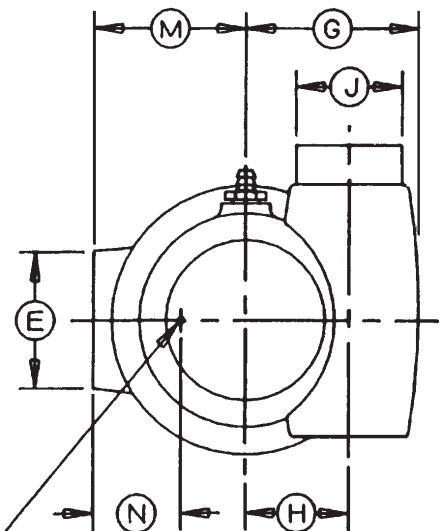
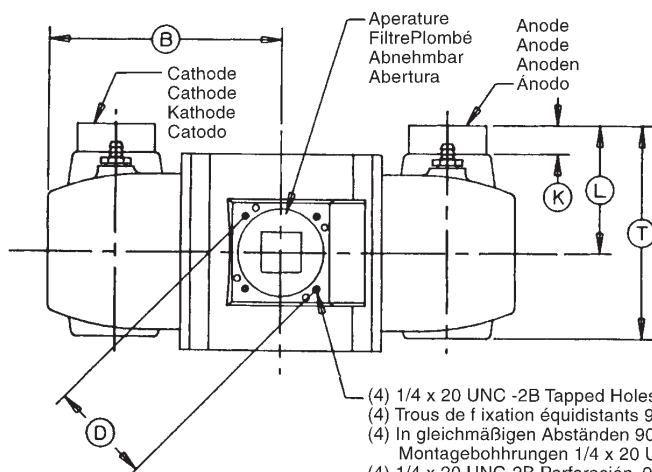
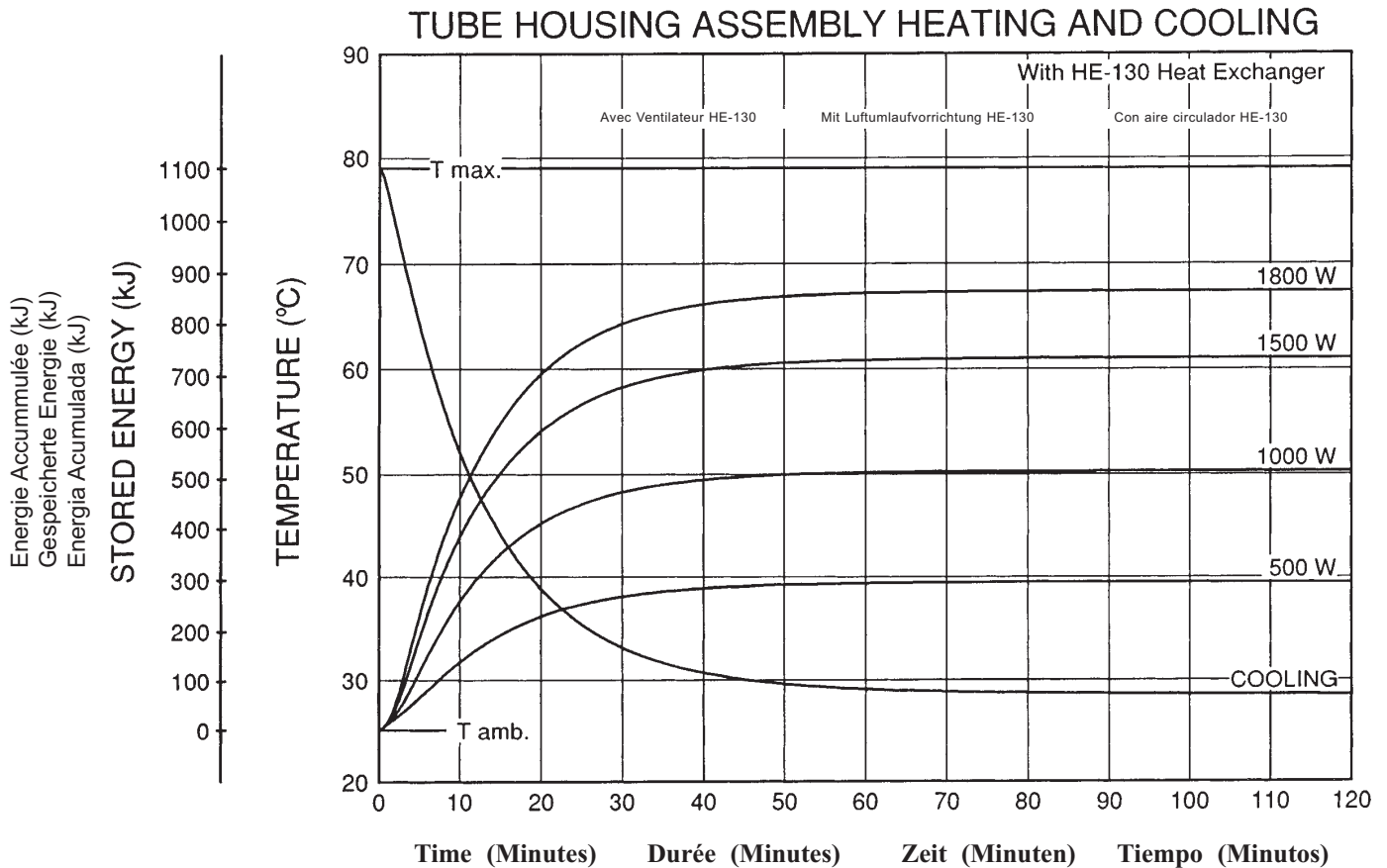
**Note:**

1. Heat inputs into housing include tube power, filament power, and stator power.
2. Heating curves based on no restrictions of natural convection around tube housing assembly.
3. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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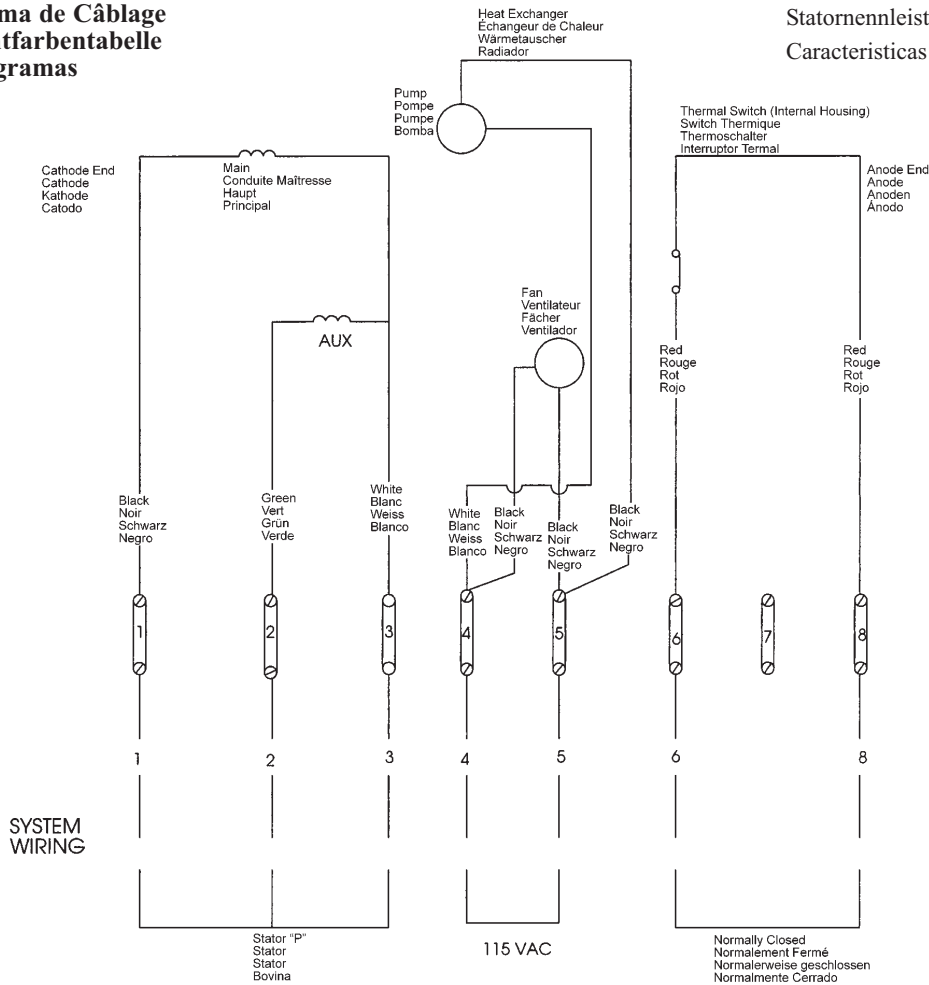
<b>Stator Type:</b> GE or SARC		
<b>Stator Coil Resistance:</b>		
Black to White	23.5 Ohms ±15%	
Green to White	23.5 Ohms ±15%	
<b>Starter Voltage:</b>		
	<b>Start</b>	<b>Run</b>
50/60 Hz	220 VAC	75 VAC
150/180 Hz	385 VAC	85 VAC
<b>Time to Full Speed:</b>		
60 Hz	0 - 3400 RPM	3.0 Sec.
180 Hz	0 - 7500 RPM	4.4 Sec.
<b>X-Ray Tube Assembly:</b>		
S-270/B-150H	IEC 60601-2-28	
(Test based on "Maximum Continuous Anode Input Rate")		

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