Tubes Radiogènes à Anode Tournante  
Drehanoden - Röntgenröhre  
Tubos de Rayos - X con Ánodo Giratorio

Note: Document originally drafted in the English language.

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Manufactured by Varian Medical Systems  
Fabrique par Varian Medical Systems  
Hergestellt von Varian Medical Systems  
Fabricado por Varian Medical Systems

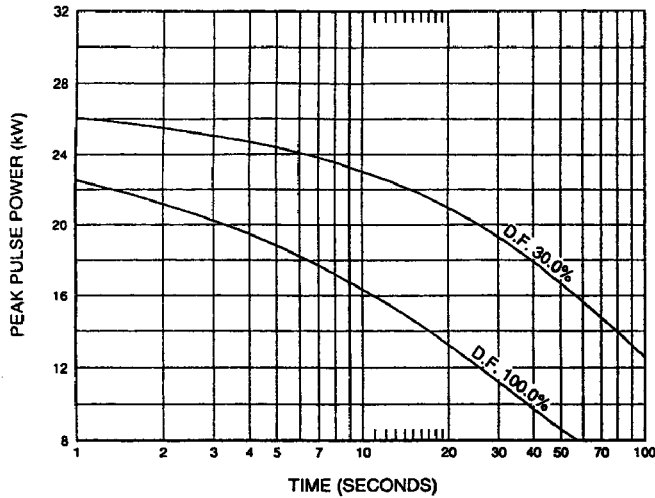
Specifications subject to change without notice.  
Spécifications susceptibles d'être modifiées sans préavis.  
Technische Daten ohne Gewähr.  
Especificaciones sujetas a cambio sin previo aviso.

### 3 Ø Constant Potential


50 Hz

Abaques d'Expositions CEI 60613  
Belastungskurven IEC 60613  
Diagramas de Exposición IEC 60613

**Nominal Focal Spot Size - 0.8**


**Nominal Focal Spot Size - 1.5**

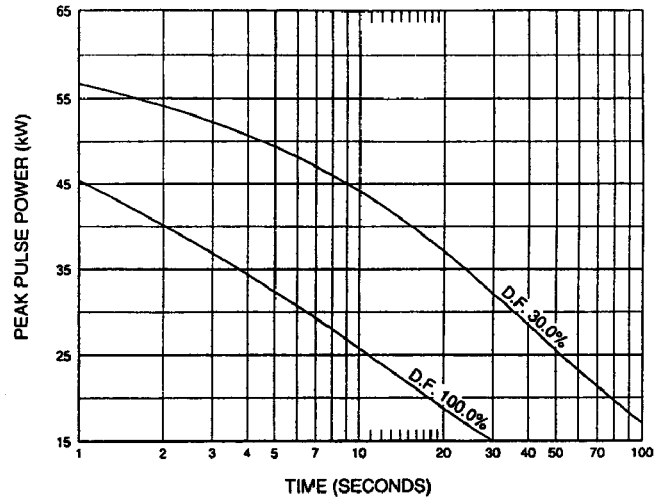
180 Hz

**Nominal Focal Spot Size - 0.8**

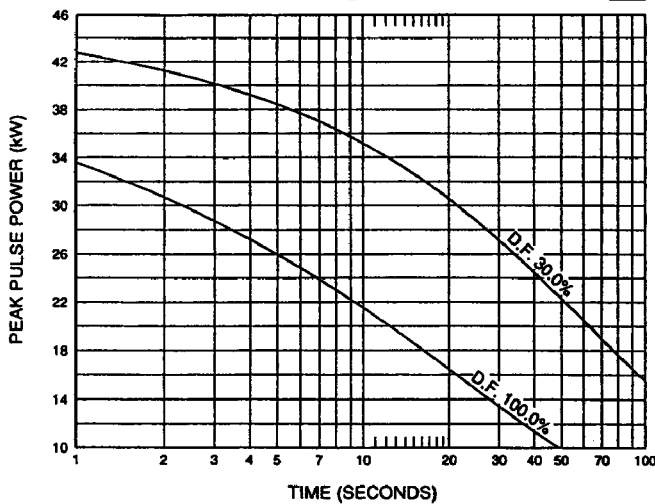**Nominal Focal Spot Size - 1.5**

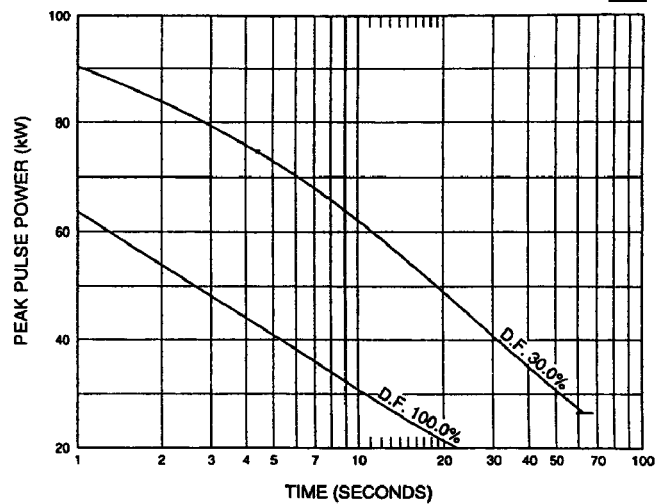**Note:**  
Rating charts reflect maximum tube performance. Tube operation is ultimately limited by system software.

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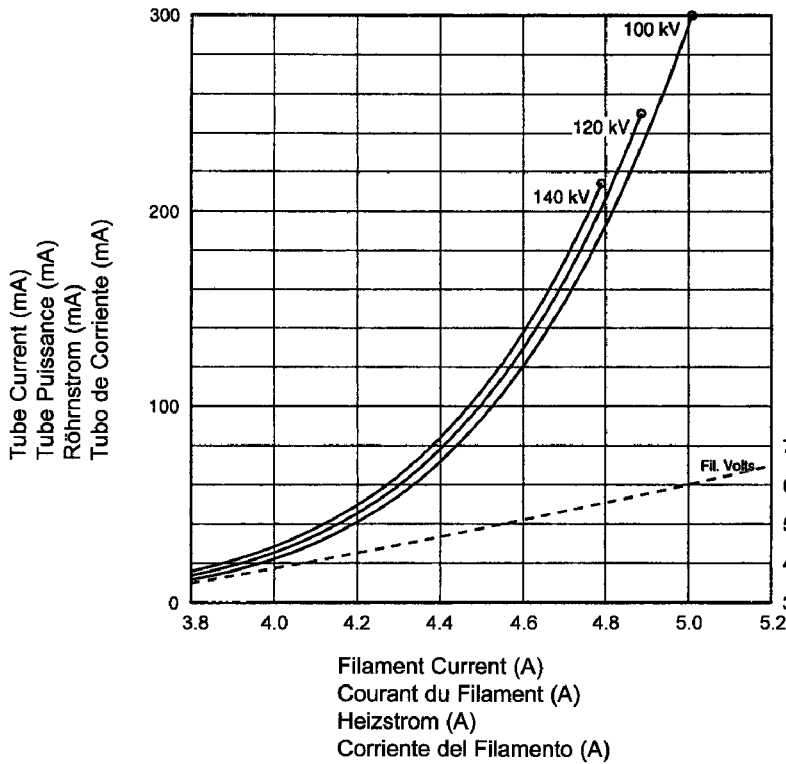
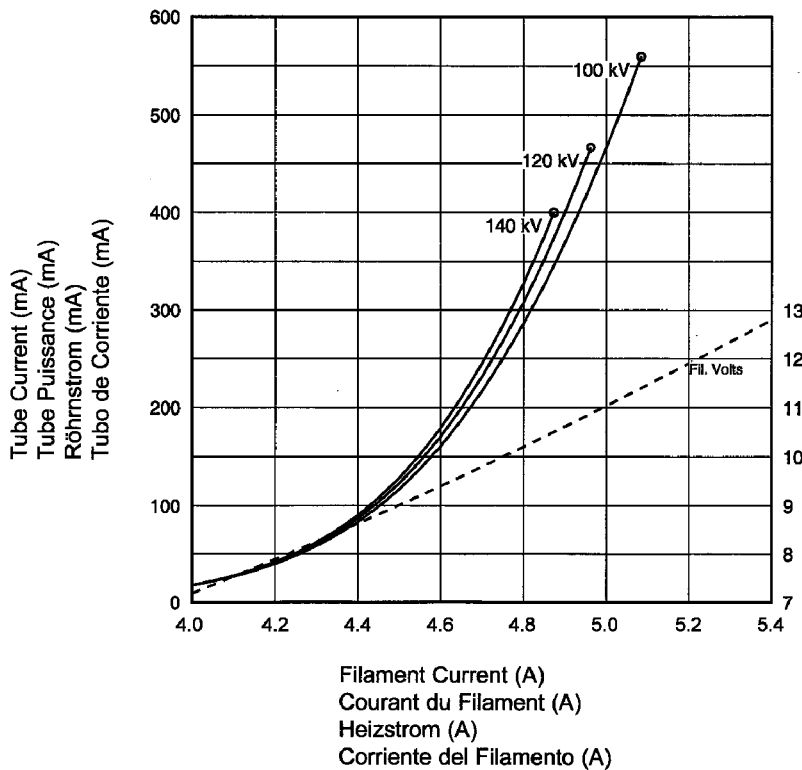
**Le Gaine B160H**

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**Encaje de B160H**

Maximum Peak Voltage .....	150 kV
Anode to Ground .....	75 kV
Cathode to Ground .....	75 kV
Maximum X-ray Tube Assembly Heat Content .....	1.1 MJ (1.5 MHU)
Maximum Continuous Heat Dissipation (Includes stator heat) .....	1,490 W (2,080 HU/sec)
Maximum Heat Exchanger Dissipation .....	1,800 W (2,540 HU/sec)
Focal Point Position (Central Ray) Within 1mm (X,Y Direction from the center of radiation port.)	
X-Ray Tube Assembly Permanent filtration .....	1.0 mm Al IEC 60522
Loading Factors for Leakage Radiation .....	150 kV, 4 mA
Federal Standard High Voltage Cable .....	72
Ambient Air Temperature Limits for Operation .....	5°C to 40°C
Temperature Limits for Storage and Transport .....	-20°C to +70°C
Humidity .....	+10% to +90%
Weight - Housing .....	23 kg (50.7 lbs)
Heat Exchanger .....	11.5 kg (25.3 lbs)
IEC Classification .....	Class I, Type B
Safety Devices: Thermal Switch Normally Closed Contact .....	Opening at 85°C
Flow Switch - Normally Open contact Contacts close with adequate oil flow.	
Filament Frequency Limits .....	50 HZ - 400 HZ
Power Supply .....	DC

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


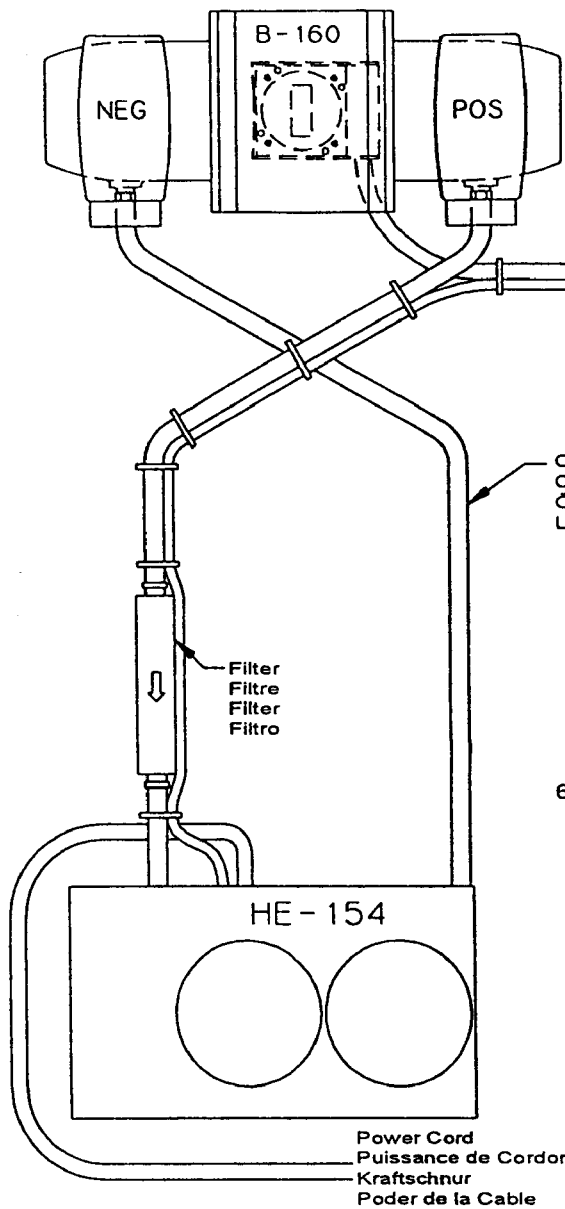
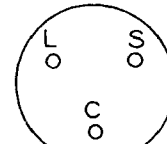
(As viewed, looking down into the receptacle)

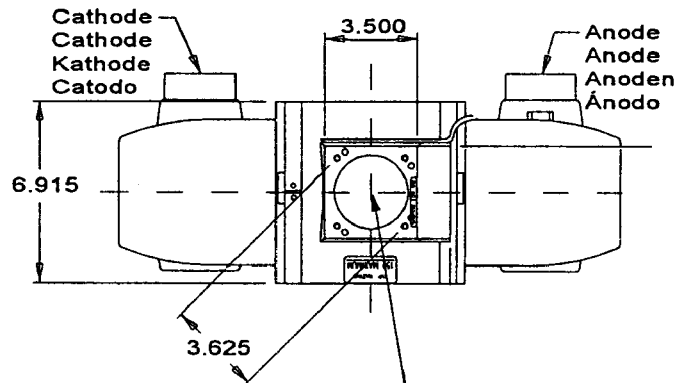
Receptacle key  
Clef du Receptacle  
Hochspannungsbuchsen  
Llava del Receptaculo

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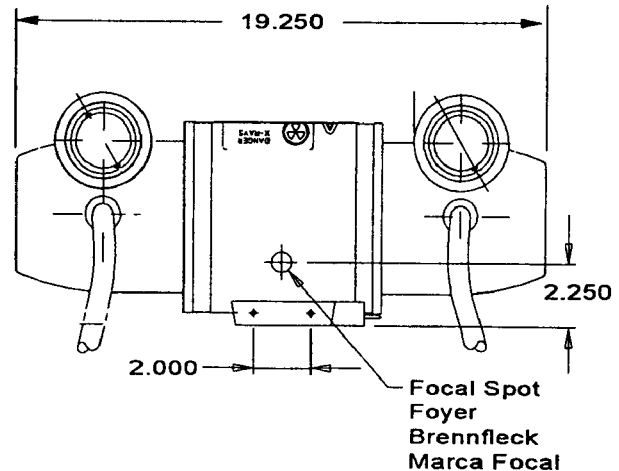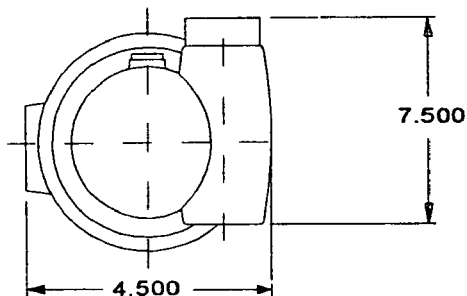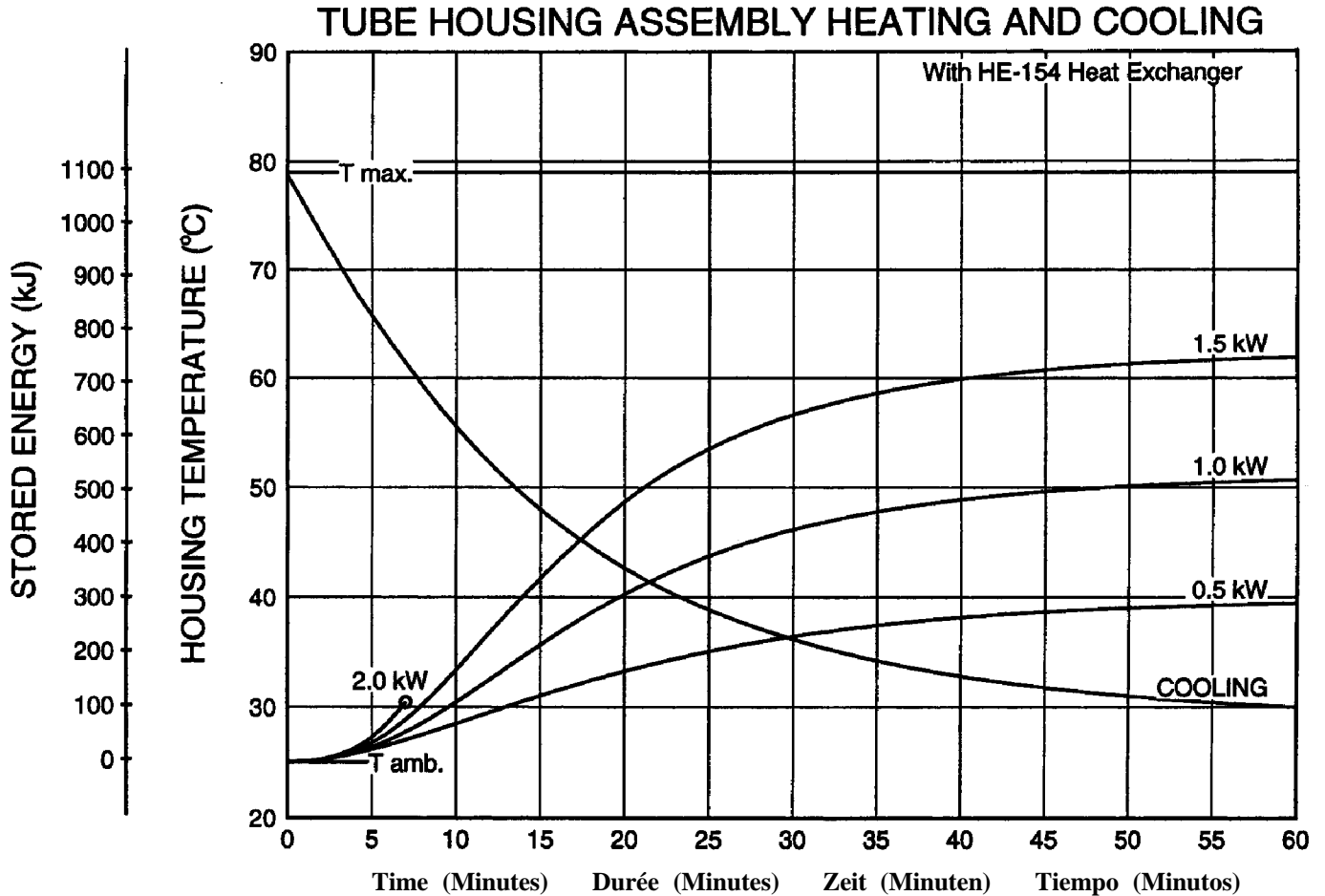**Note:**

1. Heat inputs into housing include tube power, filament power, and stator power.
2. Heating curves based on no restrictions of natural convection around tube housing assembly.
3. Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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
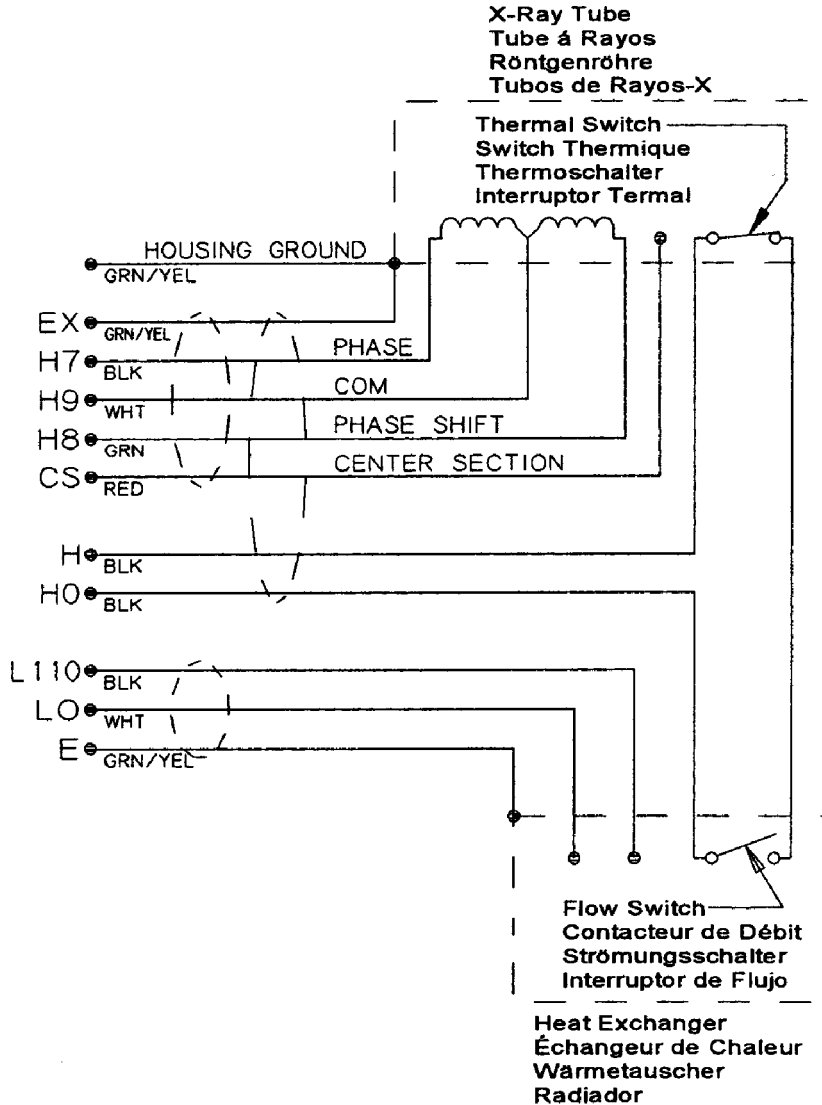
<b>Stator Type:</b> "R" Stator		
<b>Stator Coil Resistance:</b>		
Black to White	16.0 Ohms ±15%	
Green to White	50.0 Ohms ±15%	
<b>Starter Voltage:</b>	<b>Start</b>	<b>Run</b>
50/60 Hz	265 VAC	75 VAC
150/180 Hz	600 VAC	115 VAC
<b>Time to Full Speed:</b>		
50/60 Hz	0 - 2700 RPM	6.3 Sec.
150/180 Hz	0 - 8000 RPM	6.3 Sec.
<b>X-Ray Source Assembly:</b>		
S-780/B160H	IEC 60601-2-28	

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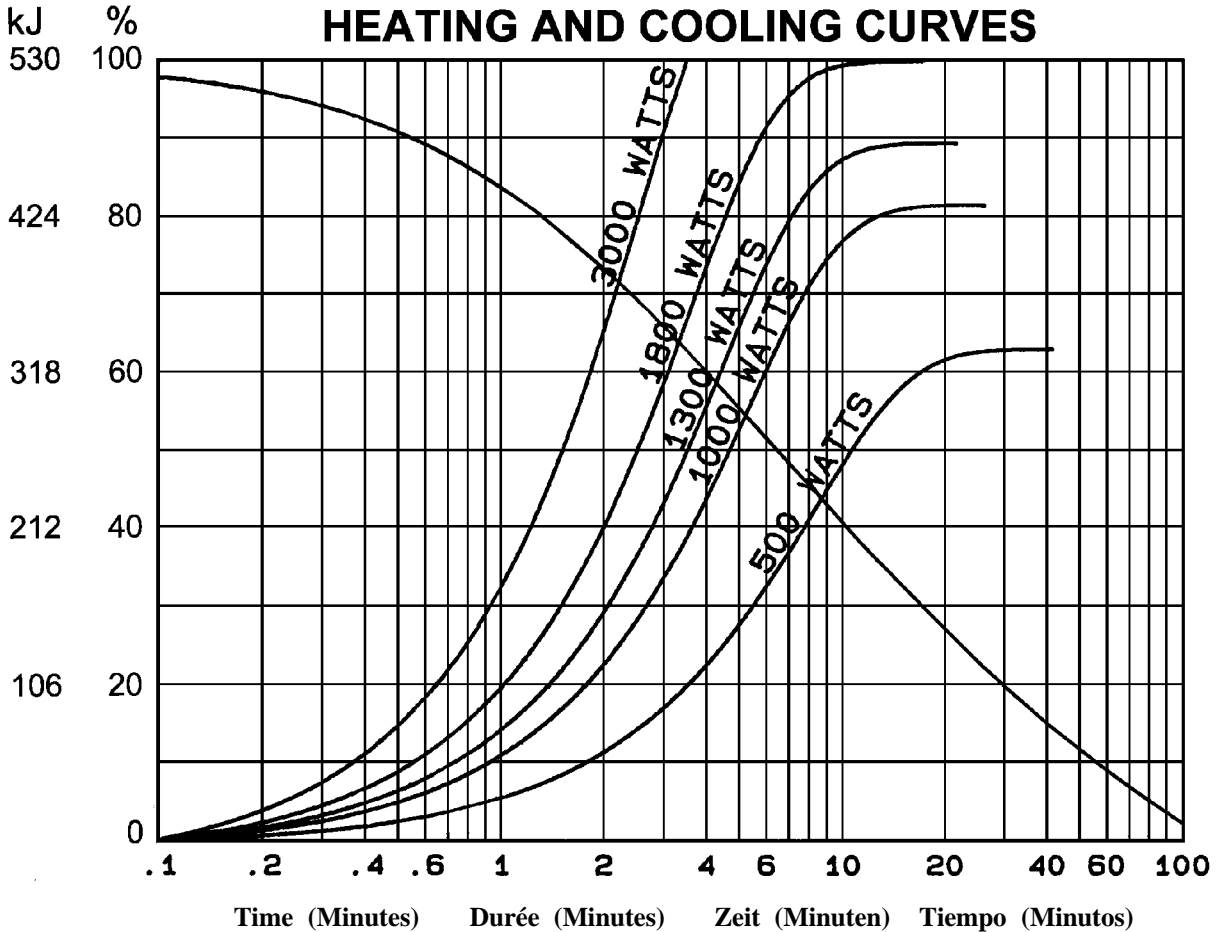**Note:**

Heating and cooling curves reflect maximum tube performance. Tube operation is ultimately limited by system software control.

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